




Size charts

Foot wrap		Measurements in cm	I	II	III	IV	V
			x-small	small	medium	large	x-large
		<b>Circumference cA</b> (over forefoot)	22 – 24	25 – 27	28 – 30	31 – 34	35 – 40
		<b>Length A</b>					
	normal		16.5	18	19.5	21	23
	long		18.5	20	21.5	23	25

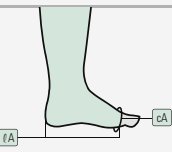
  

Calf wrap		Measurements in cm	I	II	III	IV	V
			x-small	small	medium	large	x-large
		<b>Circumference cC</b> (over mid calf)	35 – 45	40 – 52	48 – 60	55 – 65	60 – 70
		<b>Circumference cB</b> (over ankle)	20 – 27	25 – 32	30-38	37-44	45-52
	<b>Length AD</b>						
	normal		37	37	42	42	44
	long		42	42	47	47	49

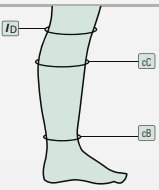
  

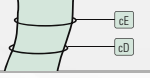Knee wrap		Measurements in cm	I	II	III	IV	V
			x-small	small	medium	large	x-large
		<b>Circumference cE</b> (around the knee)	40 – 55	50 – 65	60 – 75	70 – 85	85 – 100
		<b>Circumference cD</b> (over top of calf)	35 – 45	40 – 52	48 – 60	55 – 65	60 – 70


  

Thigh wrap		Measurements in cm	I	II	III	IV	V
			x-small	small	medium	large	x-large
		<b>Circumference cG</b> (largest point)	50 – 65	60 – 75	70 – 80	75 – 90	85 – 100
		<b>Circumference cF</b> (10 cms above E)	45 – 52	50 – 59	60 – 66	67 – 73	74 – 80
	<b>Length FG</b>						
	normal		23	23	23	23	23
	long		29	29	29	29	29


Arm wrap		Measurements in cm	I	II	III	IV
			x-small	small	medium	large
		<b>Circumference cG</b> (over upper arm)	25 – 30	31 – 35	36 – 41	42 – 47
		<b>Circumference cE</b> (over elbow)	23 – 27	28 – 32	33 – 37	38 – 43
	<b>Circumference cC</b> (over wrist)	14 – 16	17 – 19	20 – 22	23 – 25	
	<b>Length CG</b>					
	normal		42 – 47	42 – 47	42 – 47	42 – 47
	long		46 – 50	46 – 50	46 – 50	46 – 50

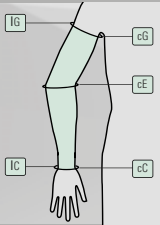Hand wrap		Measurements in cm	I	II	III	IV
			x-small	small	medium	large
		<b>Circumference cA</b> (over knuckles)	< 18	18 – 20	20 – 22	> 22

# SPECIAL-PURPOSE OEDEMA PRODUCTS
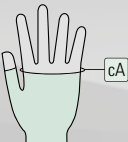
## Juzo® ACS light – Adjustable compression system

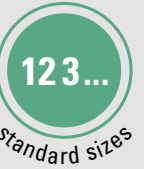

EN - 1403003/1/1 - W481 - 05/2016

# Juzo® ACS light

## Efficient and effective adjustable compression system

This short-stretch compression wraps can be employed as an efficient supplementary treatment during the decongestion phase. The Juzo ACS light products can be adjusted for the individual wearer by means of velcro fasteners. Thanks to the incorporated lining, they are easy to use and can be applied in next to no time. This ease of application increases patient compliance. Therapeutic compression pressures up to 40 mmHg are possible for arm and leg.



### Lower extremities

The Juzo ACS light wraps for the lower extremities are compression tools with short-stretch characteristics, intended to reduce oedema in the foot, lower leg, knee and thigh. Each part can work in combination with the other! The calf and thigh wraps are left and right specific – to enable a better fit and to make them easier to don and doff. The foot bandage and knee bandage can be used for both left and right and the knee has a unique cut-out in the reverse for comfort.

### Upper extremities

The Juzo ACS light for the upper extremities are compression tools with short-stretch characteristics intended to reduce oedema in the arm and hand. Juzo ACS light arm wraps can be combined with Juzo ACS light hand wraps range. The wraps for arm are left and right specific – to enable a better fit and to make them easier to don and doff. The Juzo ACS light hand wraps are particularly suitable to treat oedema in the back of the hand.

### Features

- high working and low resting pressure
- compression levels up to 40 mmHg
- can be worn day and night
- adaptable whilst wearing for optimal compression
- easy to apply – even with limited dexterity and mobility
- patients can easily self-manage thanks to the built-in liner
- can be worn with Juzo Compression Stockings, Sleeves or Gloves and also with Juzo SoftCompress products
- various accessories available



high working pressure



durable



standard sizes

### Colour

Black

### Indications

- lymphoedema
- lipoedema
- post-traumatic oedema
- post-operative oedema

### Contra indications

#### Absolute contra indications

- moderate to severe arterial disease (ankle-arm-index < 0,7)
- reduced sensitivity of the extremities
- untreated vascular infection
- uncontrolled (decompensated) congestive heart failure
- untreated or advanced skin disease
- swelling of the extremities of unknown cause
- advanced peripheral vascular disease
- untreated open wounds
- erysipelas

#### Relative contra indications

- peripheral neuropathy with diabetes mellitus
- congestive heart failure

### Product range

Model	Article number	Standard sizes I-V	Standard sizes I-IV	Uni size
<b>Foot wrap</b> Length: normal 16.5 – 23 cm long 18.5 – 25 cm	J701LI	•		
<b>Calf wrap</b> Length: normal 37 – 44 cm long 42 – 49 cm Available for left or right	J702LI	•		
<b>Knee wrap</b>	J703LI	•		
<b>Thigh wrap</b> Length: normal 23 cm long 29 cm Available for left or right	J704LI	•		
<b>Arm wrap</b> Length: normal 42 – 47 cm long 46 – 50 cm Available for left or right	J705LI		•	
<b>Hand wrap</b>	J706LI		•	
<b>Extender</b>	J716LI			•

### Material composition

85 % Polyamide  
15 % Elastane

### Storage instructions

Always store Juzo ACS light products at room temperature and away from direct sunlight.

### Washing and care instructions

Before washing, please ensure that any velcro fastenings are closed. Wash Juzo ACS light products at least once a week, if required more often. Juzo ACS light products can be washed in the washing machine at 30 °C using a laundry detergent for delicate fabrics. Please do not use any fabric softener. Do not bleach. Do not dry in a clothes dryer. To reduce the drying time, place Juzo ACS light products on a thick terry towel, roll it up tightly and squeeze it firmly. Do not leave in the towel and do not dry on a radiator or in the sun. Juzo ACS light products must not be ironed. Juzo ACS light products must not be cleaned chemically.



### Juzo® SoftCompress

Juzo SoftCompress Products are employed as an effective treatment aid for compression bandaging in oedema treatment. Using the comfortable bandages and accessoires can significantly improve the treatment outcome. The comfortable Juzo SoftCompress Bandages can be worn under wraps, short-stretch bandages or medical compression stockings, and provide additional compression, cushioning and an ideal pressure distribution. The Juzo SoftCompress products are especially skin-friendly and breathable. The material is washable up to 60 °C.

For further information regarding the Juzo SoftCompress product range please see the Juzo brochure SoftCompress – Special products for oedema therapy.