


SPECIAL-PURPOSE GARMENTS

For lymphological compression therapy

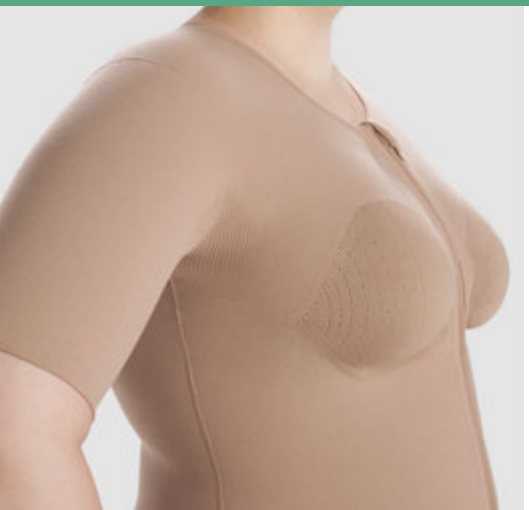

Our Customer Service department – always there for you

Customer service is a top priority at Juzo – and has been for more than 100 years. Our passionate, reliable and flexible staff is happy to advise you on any aspect of compression garments. Your contact persons are knowledgeable in all areas of our quality products and are glad to assist you with any questions or issues you might have.

Our Customer Service department offers ...

- general information
- individual product advice
- order acceptance
- feedback on the status of your order
- acceptance of alteration or repair orders
- advice for specialist questions regarding flat knitting and oedema treatment
- support for complex and complicated flat knitted garments



Edith Männer
Head of Customer Service



Markus Daferner
Technical Service

You can reach us by phone

Monday to Friday.

Please feel free to send us an e-mail or a fax.

Juzo Customer Service

International

Office hours 7:00 am – 6:00 pm

Phone +49 (0)8251 901 300

Fax +49 (0)8251 901 321

E-mail order@juzo.com

Juzo Customer Service

UK

Office hours 9:00 am – 5:00 pm

Phone +44 (0)1382 82 66 20

Fax +44 (0)1382 82 66 41

E-mail sales@juzo.co.uk




More than 100 years of experience in compression therapy

The history of the family-run business Juzo extends back to the year 1912. It was in that year that Julius Zorn GmbH (Juzo) was founded in Zeulenroda. This small city in the southeast of Thuringia was already regarded at the time as the elastic stocking manufacturing capital – a place where a high level of know-how and experienced employees came together. The hosiery industry in Zeulenroda has its roots as far back as 1750 and gained an international

reputation in the 19th century. In the year 1919 Juzo revolutionised its manufacturing processes: for the first time ever the stockings were now knitted rather than woven. The demand for these novel products increased steadily thereafter. By the 30s Juzo was already exporting its compression products to many countries in the world. After the Second World War, the grandson of the founder Julius Zorn settled in Aichach and developed the company Juzo

into an international business. Juzo introduced seamless compression stockings in 1951, and in 1962 began manufacturing the first latex-free compression stockings. Thanks to its consistent focus on quality and further developing its products, Juzo has been successfully growing its international business and currently employs around 1000 people all over the world. The manufacturing facilities at the company's site in Aichach are the most modern of their kind.

How to quickly choose the right product

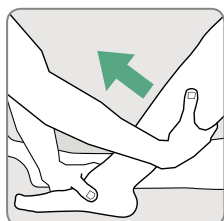

04 // The effective oedema therapy



05 // Versatile compression products




06 // Juzo Expert and Juzo Expert Strong


Product overview




07 // Foot-toe-portions

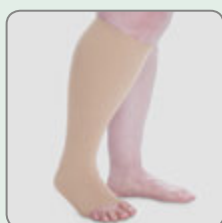


08 // Stockings (AD)

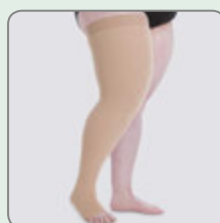

10 // Stockings (AG)

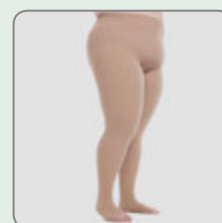


13 // Pantyhose (AT)

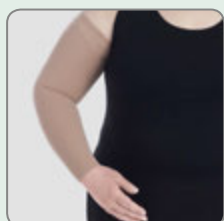


16 // Sleeves




18 // Gloves

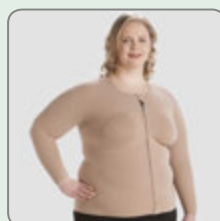

20 // Thorax segments

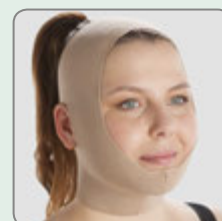

22 // Head segments

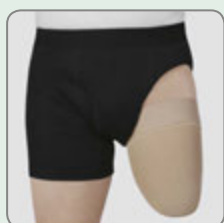

23 // Stump shrinkers (arm and leg)




25 // SoftCompress



25 // Lymph pads



26 // Donning and doffing aids

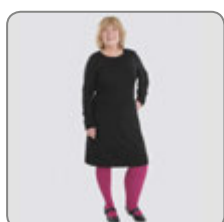

27 // Practical tips for the supplier

The effective oedema therapy

The building blocks

Effective oedema therapy is based on the building blocks of the Complex or Combined Physical Decongestive Therapy:

- Manual Lymphatic Drainage (MLD)
- compression therapy
- skin care
- sport and exercise therapy
- education and training in individualised self-therapy

These building blocks will be implemented in two phases:

Phase 1: Decongestion phase

The transport capacity of the lymphatic system is first increased with the aid of Manual Lymphatic Drainage (MLD) to enable the protein-rich oedema fluid to be mobilised and transported away. Compression is then applied to the decongested extremity using compression dressings or bandages (e.g. Juzo SoftCompress). This prevents the fluid from flowing back and reduces connective tissue proliferation.

Phase 2: Maintenance phase

During the second phase, the number of MLD sessions is reduced and the compression dressings or bandages are replaced by flat knitted compression garments (e.g. Juzo Expert or Juzo Expert Strong) in order to optimise the therapeutic results and maintain them over the long-term. Consistent skin care during this phase protects against drying and minimises the risk of infections. Physical activity and decongestive exercises aid in stimulating lymphatic function.



Versatile compression products

The right choice for an optimum treatment outcome

Compression with lymphological bandages or wraps



soft material

Juzo SoftCompress

Special products for oedema therapy

Using Juzo SoftCompress products can significantly improve the treatment outcome. The comfortable bandages can be worn under short-stretch bandages or compression stockings or wraps. They provide an ideal pressure distribution.

For further information regarding the Juzo SoftCompress product range please see the brochure "Juzo SoftCompress – Special products for oedema therapy".



high working pressure

Juzo ACS light

Adjustable compression system

The short-stretch compression wraps can be employed as an efficient supplementary treatment during the decongestion phase. The Juzo ACS light products can be adjusted for the individual wearer by means of velcro fasteners. Thanks to the incorporated lining, they are easy to use and can be applied in next to no time.

For further information regarding the Juzo ACS light product range please see the folder "Juzo ACS light – Special products for oedema therapy".

Compression with flat knitted compression garments



UV protection



flat knitting

Juzo Expert

Pleasantly soft for tailored treatment

Thanks to its exceptional softness and bi-directional elasticity, Juzo Expert provides the greatest possible freedom of movement and is very comfortable to wear. This special fabric is ideally suited for full-body garments and, thanks to its certified UV protection factor of 80 (18 – 32 mmHg, CCL 1 – 2), is also ideal for use in scar therapy.



durable



flat knitting

Juzo Expert Strong

Reliable and strong products for lymphology

Thanks to its high therapeutic pressure and pleasant massaging effect, Juzo Expert Strong is especially suitable for severe lymphological disorders. The firm and strong fabric provides professional support in the treatment of pronounced lymphoedema and lipoedema.

Compression fabrics with silver

Patients with lymphological or phlebological disorders, or who are suffering from burns or scalds, are frequently affected by bacterial skin infections. Compression therapy fabrics with a knitted-in antibacterial silver thread – for example Juzo Expert Silver and Juzo Expert Strong Silver – provide effective protection against these kinds of skin disorders. The proportion of silver in the textile, optimised to protect the skin flora, restores the balance of the inflammation-prone and irritated skin thereby protecting against further infection. The silver ions released by the thread produce an antibacterial effect that combats a wide spectrum of bacteria, for example *Staphylococcus aureus*, which are more frequently present on damaged skin. Wearing a silver-containing compression fabric reduces both the bacteria on the skin and itchiness, thereby avoiding skin disorders.



antibacterial

Compression fabrics with cotton

Juzo Expert Cotton and Juzo Expert Strong Cotton are especially soft and gentle on the skin thanks to their high cotton content. Because they cover the skin entirely with pure natural fibre, cotton garments are especially recommended for sensitive skin. These fabrics are characterised by their high level of wearing comfort and comfortable skin environment. They are also easier to put on and take off due to their softness.



with cotton

Juzo Expert and Juzo Expert Strong

Compelling product features

Versatile fabrics for effective compression therapy

Medical compression stockings from Juzo guarantee correct pressure values and a well-defined pressure gradient. This thicker fabric with seams offers additional benefits, for example a massage effect that promotes lymph drainage, and does not yield to oedema such as lymphoedema or lipoedema. In conjunction with movement and the associated muscular contractions, it produces a high therapeutic pressure that ensures an optimum compression of the decongested tissue. The flat knitted products Juzo Expert and Juzo Expert Strong are knitted stitch by stitch to precisely match the body measurements of the patient. Juzo garments are made from various combinations of materials, for example incorporating cotton or silver, thereby making them suitable for a wide variety of requirements.



- optimum wearing comfort for the patient thanks to the especially flat and elastic seams
- patient mobility is maintained thanks to the homogeneous, soft and smooth material
- a double-layered covering of the compression fibres with textile yarns ensures a high level of skin compatibility
- the compression garment is suitable for everyday wearing thanks to the use of latex-free, sunfast and ointment proof, durable and breathable materials
- the fine-meshed surface structure of the fabric combined with optionally incorporated fasteners (hook, zipper or velcro) allow easy donning and doffing
- optimum sun protection with a certified UV-protection factor of 80 for Juzo Expert CCL 1 and 2

Colours*


Juzo Expert and Juzo Expert Strong (CCL 1 – 3)



Trend colours* and Batik Collection

Juzo Expert and Juzo Expert Strong (CCL 1 – 3)

 Trend colours See www.juzo.com/trend-colours for the current trend colours.

 Batik Collection All colours (except Almond) are available in tie dye in combination with white or black for compression stockings, pantyhose and sleeves.

Juzo quality – strictly controlled and consistently good



Oeko-Tex Standard 100



Quality seal RAL



UV Standard 801



ISO certification

For comprehensive information on Juzo's quality standards, visit our website.

*CCL 4 and garments in Silver and Cotton are only available in colour Almond.

Product overview

Compression foot-toe-portions



MODELS



EXTRA OPTIONS

1 Toes

Individual extra equipment

Open toes	Closed toes	Little toe (5) open Diagonal opening	Little toe (5) open Straight opening	Seams on the outside
		• standard		<ul style="list-style-type: none"> • exceptionally flat seam on the skin • suitable for sensitive skin • also available with open toes

Accessories

Lymphpad Line	Lymphpad Square	Adhesive Lotion	Lymph Lotion	Special Detergent	Special Gloves
<ul style="list-style-type: none"> • stimulates microcirculation in the tissue 	<ul style="list-style-type: none"> • stimulates microcirculation in the tissue 	<ul style="list-style-type: none"> • special-purpose adhesive lotion for securing of compression fabrics 	<ul style="list-style-type: none"> • skin care lotion with 4 % urea • in the treatment of lymphoedema and lipoedema and also venous afflictions 	<ul style="list-style-type: none"> • for medical compression garments and elastic fabrics • highly concentrated and economical to use 	<ul style="list-style-type: none"> • rubberised palm • makes donning of compression garments easier

Compression stockings (AD)



MODELS

AD



AB



BD



EXTRA OPTIONS


1 Toes

Ball stub for open toes



- for oedema on the bridge of the foot
- angled, distal end

Ball stub for closed toes

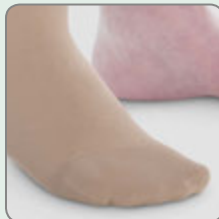

- for oedema on the bridge of the foot
- angled, distal end

Open toes




Closed toes



- especially elastic
- high wearing comfort

Sewn-on toe portion

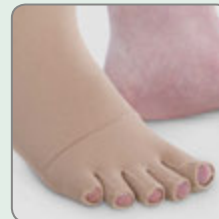

- no overlapping of stocking and toe part
- also available with 5th toe open

2 Border / fastening

Overheight / Overheight max



- lateral, angled opening
- optimised pressure decrease in the proximal direction

Top border



- additional fastening possible if required (e.g. adhesive lotion)

Silicone border (5 cm wide)



- 3.5 cm wide available on request
- silicone beads ensure high level of stability and breathability

Decorative silicone border (5 cm wide)



- silicone beads ensure high level of stability and breathability

Lace border (5 cm wide)



- better hold thanks to circular silicone bands

Elastic border (3.5 cm wide)



- stable border
- without silicone

Individual extra equipment

Additional instep stitching (Y-stitching/90° foot)



- reduces folding in the instep area thanks to an anatomically knitted heel

Silicone stopper (Adhesive borders)



- possible placement: lateral at the outer side

Tricot lining sewn-in at "cY"



- for patients with sensitive skin in the instep area
- in case of skin irritation on request with silver

Incorporated pressure pads by Dr. Rotter



- improvement of compression in the Bisgaard's region
- in case of skin irritation on request with silver
- malleolar pads made of silicone also available as an alternative

Internal sewn-in silicone border (only with overheight)



- more stability and hold
- 3/4 internal band also available on request
- combinable with above attached silicone border

Incorporated pocket for pressure pad




- individual according drawing on request
- in case of skin irritation on request with silver

Seams on the outside




- particularly flat seams on the skin
- suitable for sensitive skin

Zipper

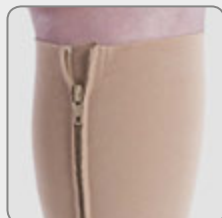

- standard position in the seam
- individual location on request

Silicone coating at the sole of foot

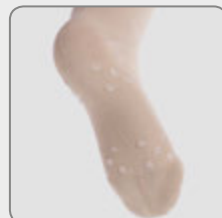

- anti-slip coating

Accessories

Pressure pad by Dr. Rotter (carbason moulded foam)



- improvement of compression in the Bisgaard's region
- in case of skin irritation on request with silver

Malleolar pad made of silicone



- improvement of compression in the Bisgaard's region
- soft and flat
- also available with textile elastic cover / cover containing silver

Lymphpad Line



- stimulates microcirculation in the tissue

Lymphpad Square



- stimulates microcirculation in the tissue

SoftCompress Malleo Pads



- improvement of compression in the Bisgaard's region

Liner Stocking Silver



- antibacterial thanks to knitted-in silver thread
- securing of wound dressings or pressure pads

Lymph Lotion



- skin care lotion with 4 % urea
- in the treatment of lymphoedema and lipoedema and also venous affections

Arion Easy-Slide Leg



- donning aid for compression stockings / pantyhose with open toes

Arion Sim-Slide



- donning and doffing aid for compression stockings / pantyhose with open toes

Arion Magnide



- donning aid for compression stockings / pantyhose with closed toes

Arion Easy-Slide Leg + Easy-Off



- donning and doffing aid for compression stockings / pantyhose with open toes

Special Gloves



- rubberised palm
- makes donning of compression garments easier

Compression stockings (AG)



MODELS

AG



BG



EXTRA OPTIONS

1 Toes

Ball stub for open toes



- for oedema on the bridge of the foot
- angled, distal end

Ball stub for closed toes



- for oedema on the bridge of the foot
- angled, distal end

Open toes



Closed toes



- especially elastic
- high wearing comfort


Sewn-on toe portion



- no overlapping of stocking and toe part
- also available with 5th toe open


2 Border / fastening

Overheight / Overheight max

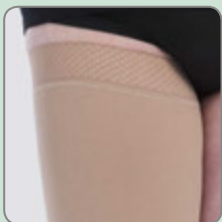


- lateral, angled opening
- optimised pressure decrease in the proximal direction

Overheight at the front of the leg

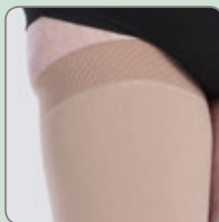

- enhanced adhesion for larger ventral thighs or oedema

Top border

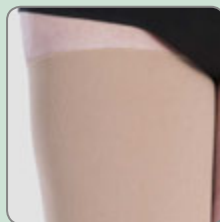

- additional fastening required (e.g. adhesive lotion)

Silicone border (5 cm wide)



- silicone beads ensure high level of stability and breathability

Decorative silicone border (5 cm wide)






- silicone beads ensure high level of stability and breathability




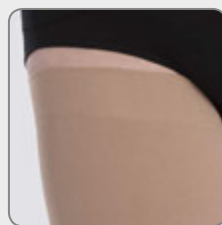




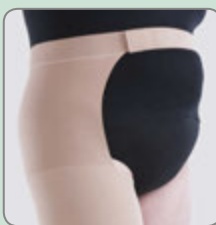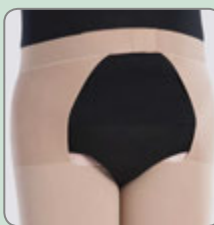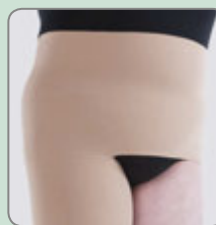



Lace border (5 cm wide)



- better hold thanks to circular silicone bands

<p>Hip attachment (single)</p>  <ul style="list-style-type: none"> • for a better hold additional silicone border on request • can be worn on the left or right side 	<p>Hip attachment to be worn as a pair</p>  <ul style="list-style-type: none"> • to open at the front and back • to open with a belt (front only) on request • for a better hold additional silicone border on request 	<p>Sewn-on body part</p>  <ul style="list-style-type: none"> • alternative fastening option with compression in the stomach area
--	---	--

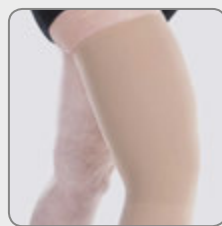Individual extra equipment
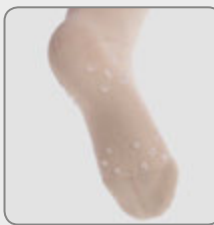
<p>Additional instep stitching (Y-stitching/90° foot)</p>  <ul style="list-style-type: none"> • reduces folding in the instep area thanks to an anatomically-knitted heel 	<p>Anatomically flexed form at "cE"</p>  <ul style="list-style-type: none"> • 30° or 50° angle available • for predominantly angled knee position 	<p>Silicone stopper (Adhesive borders)</p>  <ul style="list-style-type: none"> • placement options on thigh: lengthwise at the front, lateral at the outer side 	<p>Internal sewn-in silicone border (only with overheight)</p>  <ul style="list-style-type: none"> • more stability and hold • 3/4 internal border also available on request • combinable with above attached silicone border 	<p>Tricot lining sewn-in at "cY"</p>  <ul style="list-style-type: none"> • for sensitive skin or skin folds in the arch / instep area • in case of skin irritation on request with silver 	
<p>Tricot lining sewn-in at "cE"</p>  <ul style="list-style-type: none"> • for sensitive skin in the hollow of the knee • in case of skin irritation on request with silver 	<p>Incorporated pressure pads designed by Dr. Rotter</p>  <ul style="list-style-type: none"> • improvement of compression in the Bisgaard's region • in case of skin irritation on request with silver • malleolar pads made of silicone also available as an alternative 	<p>Incorporated pocket for pressure pad</p>  <ul style="list-style-type: none"> • individual according drawing on request • in case of skin irritation on request with silver 	<p>Seams on the outside</p>  <ul style="list-style-type: none"> • particularly flat seams on the skin • suitable for sensitive skin 	<p>Porosa from "cD"</p>  <ul style="list-style-type: none"> • compression reduction from "cD" 	<p>Zipper</p>  <ul style="list-style-type: none"> • individual location on request
<p>Two bilateral spiral stays and/or patella ring</p>  <ul style="list-style-type: none"> • integrated knee support 	<p>Silicone coating at the sole of foot</p>  <ul style="list-style-type: none"> • anti-slip coating 				

Accessories

Pressure pad by Dr. Rotter (carbosan moulded foam)



- improvement of compression in the Bisgaard's region
- in case of skin irritation on request with silver

Malleolar pad made of silicone



- improvement of compression in the Bisgaard's region
- soft and flat
- also available with textile elastic cover / cover containing silver

Lymphpad Line



- stimulates microcirculation in the tissue

Lymphpad Square



- stimulates microcirculation in the tissue

SoftCompress Malleo Pads



- improvement of compression in the Bisgaard's region

SoftCompress Knee Joint Pad



- to improve the compression in the hollow of the knee

Adhesive Lotion



- special-purpose adhesive lotion for securing of compression fabrics

Lymph Lotion



- skin care lotion with 4 % urea
- in the treatment of lymphoedema and lipoedema and also venous afflictions

Special Detergent



- for medical compression garments and elastic fabrics
- highly concentrated and economical to use

Arion Easy-Slide Leg



- donning aid for compression stockings / pantyhose with open toes

Arion Sim-Slide



- donning and doffing aid for compression stockings / pantyhose with open toes

Arion Magnide



- donning aid for compression stockings / pantyhose with closed toes

Arion Easy-Slide Leg + Easy-Off



- donning and doffing aid for compression stockings / pantyhose with open toes

Special Gloves



- rubberised palm
- makes donning of compression garments easier

Compression pantyhose (AT)



MODELS

AT




Capri pants
(BT, B'T, CT)



Bermuda pants
(ET, FT)



One-legged
pantyhose

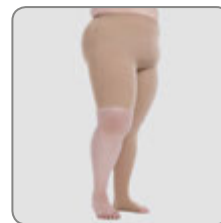

AT body parts
in general

Standard combinations:

AT	Leg parts	Body part
CCL	CCL	CCL
1	1	1
2	2	2
3	3	2
4	4	2

- every other combination available

EXTRA OPTIONS


1 Toes

Ball stub for
open toes



- for oedema on the bridge of the foot
- angled, distal end

Ball stub for
closed toes

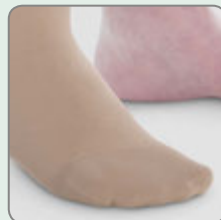

- for oedema on the bridge of the foot
- angled, distal end

Open toes



Closed toes



- especially elastic
- high wearing comfort


Sewn-on toe portion



- no overlapping of stocking and toe part
- also available with 5th toe open


2 Border / fastening

Diagonal version

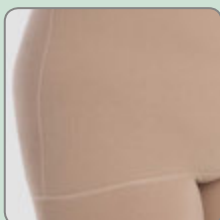


- standard
- in the case of frontal oedema development
- in the case of a flat inguinal orientation

Box type

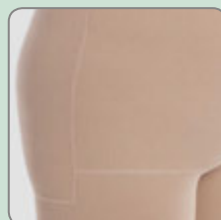


- ideal for large circumferences
- in the case of lateral oedema development
- in the case of pronounced hip and buttock shapes

Slipform

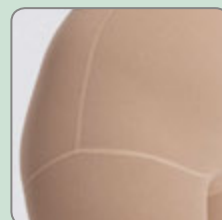


- preferred for less pronounced stages of oedema and body shapes
- in the case of a steep inguinal orientation
- in the case of a flat abdominal area
- stomach circumference: max. 140 cm

Mat Fit
(pregnancy fastening)

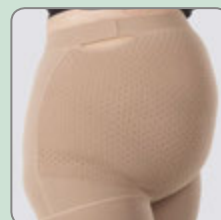


- always CCL1
- body part without compression (grows with the body)
- option: velcro fastener on the right or left
- minimum body part length: 34 cm

Top border

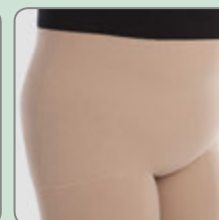

- soft border


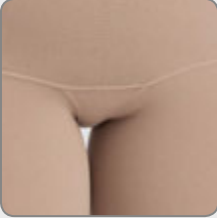

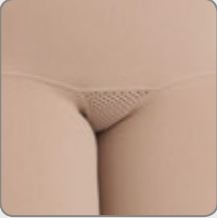


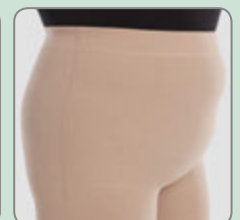











Adjustable waist band



- individually adjustable by means of elastic band with buttons

Waist band	Waist band with velcro	Silicone border (5 cm wide)	Body part with zipper	Body part with hook fastener	Body part with velcro fastener
					
<ul style="list-style-type: none"> visually flat and elastic border the edge of the knitted fabric will not fold over 	<ul style="list-style-type: none"> visually flat and elastic border individually adjustable 	<ul style="list-style-type: none"> visually flat border additional fastening with silicone beads (dots) 	<ul style="list-style-type: none"> individual positioning on request 	<ul style="list-style-type: none"> individual positioning on request 	<ul style="list-style-type: none"> individual positioning on request

Individual extra equipment

Gusset	Small gusset	Mesh gusset	Small mesh gusset	Fly	Incorporated scrotum
					
<ul style="list-style-type: none"> compressive gusset on request 	<ul style="list-style-type: none"> advantageous for large thigh circumferences compressive gusset on request 	<ul style="list-style-type: none"> promotes air circulation 	<ul style="list-style-type: none"> advantageous for large thigh circumferences promotes air circulation 	<ul style="list-style-type: none"> for men cross-mesh 	<ul style="list-style-type: none"> ideal fit in the genital area of men in combination with fly on request
Stoma opening	Open crotch	Additional instep stitching (Y-stitching / 90° foot)	Anatomically-angled shape at "cE"	Silicone stopper (Adhesive borders)	3/4 internal sewn-in silicone border
					
<ul style="list-style-type: none"> opening will be placed individually according to drawing 	<ul style="list-style-type: none"> without gusset not recommended for adipose tissue 	<ul style="list-style-type: none"> reduces folding in the instep area thanks to an anatomically-knitted heel 	<ul style="list-style-type: none"> 30° or 50° angle available for predominantly angled knee position 	<ul style="list-style-type: none"> placement options on thigh: lateral at the outer side, lengthwise at the front 	<ul style="list-style-type: none"> at thigh more stability and hold only in conjunction with body part in diagonal version
Tricot lining sewn-in at "cY"	Tricot lining sewn-in at "cE"	Incorporated pressure pads by Dr. Rotter	Incorporated pocket for pressure pad	Seams on the outside	Zipper
					
<ul style="list-style-type: none"> for sensitive skin in the instep area in case of skin irritation on request with silver 	<ul style="list-style-type: none"> for sensitive skin in the hollow of the knee in case of skin irritation on request with silver 	<ul style="list-style-type: none"> improvement of compression in the Bisgaard's region in case of skin irritation on request with silver malleolar pads made of silicone also available as an alternative 	<ul style="list-style-type: none"> individual according drawing on request in case of skin irritation on request with silver 	<ul style="list-style-type: none"> particularly flat seams on the skin suitable for sensitive skin 	<ul style="list-style-type: none"> individual location on request

Bilateral dual spiral stays and/or patella ring

Silicone coating at the sole of foot ring



- integrated knee support

- anti-slip coating

Accessories

Pressure pad by Dr. Rotter (carbosan moulded foam)



- improvement of compression in the Bisgaard's region
- in case of skin irritation on request with silver

Malleolar pad made of silicone



- improvement of compression in the Bisgaard's region
- soft and flat
- also available with textile elastic cover / cover containing silver

Lymphpad Line



- stimulates microcirculation in the tissue

Lymphpad Square



- stimulates microcirculation in the tissue

SoftCompress Malleo Pads



- improvement of compression in the Bisgaard's region

SoftCompress Knee Joint Pad



- to improve the compression in the hollow of the knee

SoftCompress Genital Pad Women



- targeted pressure transfer and cushioning of the female genital area

SoftCompress Genital Pad Men



- targeted pressure transfer and cushioning of the male genital area

Adhesive Lotion



- special-purpose adhesive lotion for securing of compression fabrics

Lymph Lotion



- skin care lotion with 4 % urea
- in the treatment of lymphoedema and lipoedema and also venous afflictions

Special Detergent



- for medical compression garments and elastic fabrics
- highly concentrated and economical to use

Arion Easy-Slide Leg



- donning aid for compression stockings / pantyhose with open toes

Arion Sim-Slide



- donning and doffing aid for compression stockings / pantyhose with open toes

Arion Magnide



- donning aid for compression stockings / pantyhose with closed toes

Arion Easy-Slide Leg + Easy-Off




- donning and doffing aid for compression stockings / pantyhose with open toes

Special Gloves




- rubberised palm
- makes donning of compression garments easier

Compression sleeves

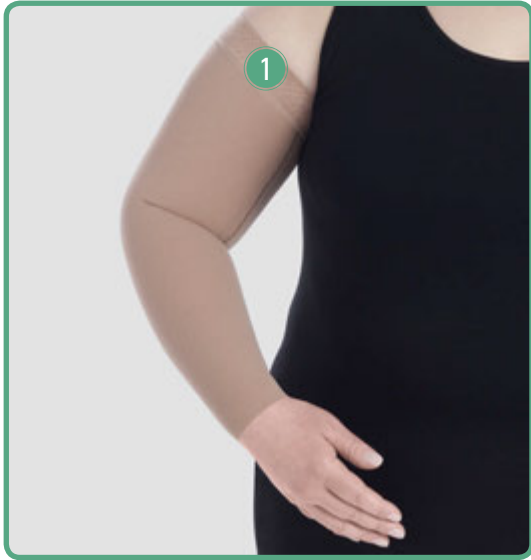


MODELS

Sleeve (CG) with overheight

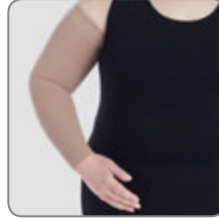

Shoulder and strap

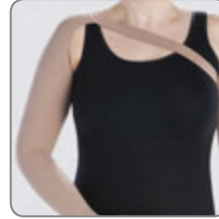

Sleeve and glove in one piece



Forearm segment (CE)



EXTRA OPTIONS


1 Border / fastening

Overheight / Overheight max at "cG"



- lateral, angled opening
- optimised pressure decrease in the proximal direction

Top border

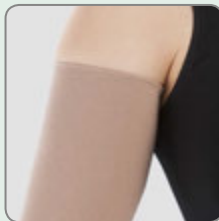

- additional fastening required (e.g. adhesive lotion)

Silicone border (5 cm wide)



- 3.5 cm wide available on request
- silicone beads ensure high level of stability and breathability

Decorative silicone border (5 cm wide)



- silicone beads ensure high level of stability and breathability

Elastic border (3.5 cm wide)




- stable band
- silicone-free

Bra fastening




- fastening option for attaching to the bra strap with velcro

Shoulder and strap

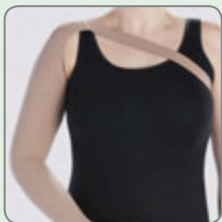


- alternative to silicone border

Bolero connection

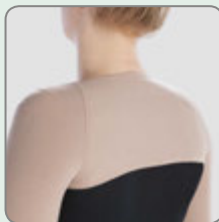

- fastening option for bilateral arm treatment
- with sleeves or short sleeves on request
- no compression in the back area

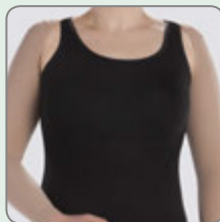

Individual extra equipment

Anatomically flexed form at "cE"




- 30° or 50° on request
- for predominantly angled elbow position

Seam at the outside of the arm at CG




- only in conjunction with anatomically flexed form 30°

Silicone stopper (Adhesive borders)

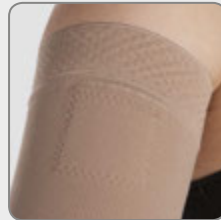


- placement options: lateral at the outer side

Internal sewn-in silicone border (only with overweight)

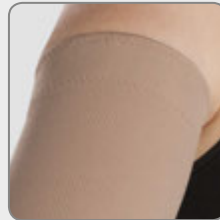

- more stability and hold
- 3/4 internal border also available on request
- combinable with above attached silicone border

Bra fastening with silicone support

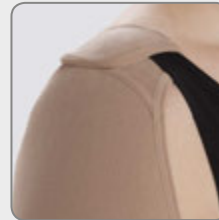

- added hold in the shoulder area

Bolero connection with silicone support



- added hold in the shoulder area

Tricot lining sewn-in at "cE"




- for sensitive skin in the crook of the arm
- in case of skin irritation on request with silver

Incorporated pocket for pressure pad



- individual according drawing on request
- in case of skin irritation on request with silver

Seams on the outside

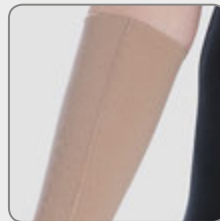

- particularly flat seams on the skin
- suitable for sensitive skin

Zipper



- individual location on request

Accessories

Lymphpad Line



- stimulates microcirculation in the tissue

Lymphpad Square



- stimulates microcirculation in the tissue

SoftCompress Elbow Joint Pad



- to improve the compression in the crook of the arm

Adhesive Lotion



- special-purpose adhesive lotion for securing of compression fabrics

Lymph Lotion



- skin care lotion with 4 % urea
- in the treatment of lymphoedema and lipoedema and also venous afflictions

Special Detergent



- for medical compression garments and elastic fabrics
- highly concentrated and economical to use

Arion Easy-Slide Arm



- smooth, two-layer material

Slippie Arm



- excellent gliding material

Special Gloves



- rubberised palm
- makes donning of compression garments easier

Compression gloves



MODELS

Glove




Long glove



Finger support



Sleeve and glove in one piece

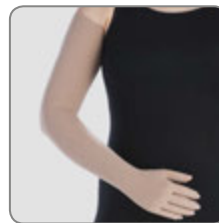

EXTRA OPTIONS

1 Finger

Glove with open fingers




Glove with closed fingers



Gauntlet with open thumb stub



Open finger support

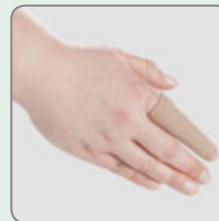

- closed thumb on request

- closed finger support on request

2 Border

Overheight


- lateral, angled opening
- optimised pressure decrease in the proximal direction

Silicone border (3.5 cm wide)



- silicone beads ensure high level of stability and breathability

Long glove

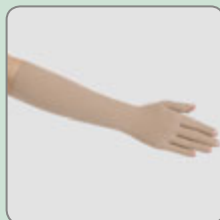


- length individually possible up to max. "cE"


Individual extra equipment

Zipper	Incorporated pocket for pressure pad	Silicone coating on the palmar surface	Tricot lining between thumb and index finger	Seams on the outside	Closed finger support with fixation
					
<ul style="list-style-type: none"> • individual location on request 	<ul style="list-style-type: none"> • pocket for standard pad • individual according drawing on request • in case of skin irritation on request with silver 	<ul style="list-style-type: none"> • improves gripability • available for open and closed fingers 	<ul style="list-style-type: none"> • for sensitive skin • in case of skin irritation on request with silver 	<ul style="list-style-type: none"> • particularly flat seams on the skin • suitable for sensitive skin 	<ul style="list-style-type: none"> • with velcro • open finger support on request

Accessories

Long pressure pad	Short pressure pad	Lymphpad Line	Lymphpad Square	SoftCompress Hand Pad	Zipper extension
					
<ul style="list-style-type: none"> • improvement of compression transfer extending through the wrist • in case of skin irritation on request with silver 	<ul style="list-style-type: none"> • improvement of compression transfer • in case of skin irritation on request with silver 	<ul style="list-style-type: none"> • stimulates microcirculation in the tissue 	<ul style="list-style-type: none"> • stimulates microcirculation in the tissue 	<ul style="list-style-type: none"> • improvement of compression transfer 	
Lymph Lotion	Special Detergent	Special Gloves	Arion Dactyna NEW		
					
<ul style="list-style-type: none"> • skin care lotion with 4 % urea • in the treatment of lymphoedema and lipoedema and also venous afflictions 	<ul style="list-style-type: none"> • for medical compression garments and elastic fabrics • highly concentrated and economical to use 	<ul style="list-style-type: none"> • rubberised palm • makes donning of compression garments easier 	<ul style="list-style-type: none"> • donning aid for compression gloves with open fingers 		

Compression segments thorax

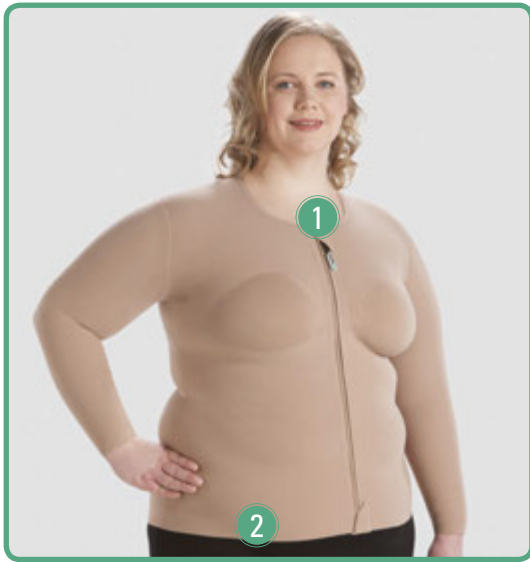

MODELS

Thorax with sleeves/short sleeves

Bodysuit



EXTRA OPTIONS


1 Closure options

Zipper



- with hook and eye underlap for easy closing
- individual location on request

Reinforced zipper

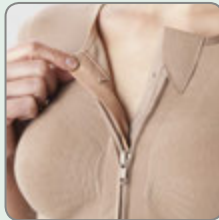


- with **reinforced hook and eye underlap** for heavy use
- individual location on request

Hook fastener



- second row of eyelets allows adjustment
- individual location on request

Reinforced hook fastener

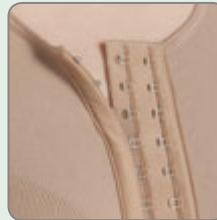

- second row of eyelets allows adjustment
- for heavy use
- individual location on request

2 Border / fastening

Silicone border (5 cm wide)




- flat opening at distal end
- with silicone beads (dots)

Waist band (5 cm wide)



- flat, elastic opening at distal end

Body fastening with hook fastener

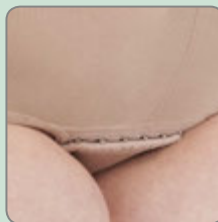

- prevents the thorax segment from sliding upwards

Body fastening with velcro



- prevents the thorax segment from sliding upwards
- alternative to hook fastener

Individual extra equipment

Cup (knitted-in bra cups)



- seamless bra cups
- lightly compressive

Pocket for breast form (epithesis)



- sewn-in pocket for breast form
- with two openings

Anatomically flexed form at "cE"



- 30° or 50° angle available
- for predominantly angled elbow position

Tricot lining sewn-in at "cE"



- for sensitive skin in the crook of the arm
- in case of skin irritation on request with silver

Sleeve with integrated zipper



- individual location on request

Incorporated pocket for pressure pad



- individual according drawing on request
- in case of skin irritation on request with silver

Stand-up collar



Seams on the outside



- particularly flat seams on the skin
- suitable for sensitive skin

Accessories

Lymphpad Line



- stimulates microcirculation in the tissue

Lymphpad Square



- stimulates microcirculation in the tissue

ScarPad extra



- silicone patch for scars

ScarPad strong



- silicone patch for scars

SoftCompress Breast Pad



- for women
- targeted compression transfer in breast and thorax area

SoftCompress Elbow Joint Pad



- to improve the compression in the crook of the arm

Zipper extension



Special Detergent



- for medical compression garments and elastic fabrics
- highly concentrated and economical to use

Arion Easy-Slide Arm



- smooth, two-layer material

Slippie Arm



- excellent gliding material

Special Gloves



- rubberised palm
- makes donning of compression garments easier

Compression segments head



MODELS

Neck-chin-bandage

Face mask



EXTRA OPTIONS

1 Head

Face mask forehead and back of head closed



- openings for eyes, nose, mouth and ears on request
- knitted-in nose on request

Face mask forehead and back of head open



- openings for eyes, nose, mouth and ears on request
- knitted-in nose on request

Face mask forehead and back of head open



- open face
- openings for ears on request

Neck-chin-bandage forehead and back of head open



- openings for ears on request

2 Closure options

Velcro fastener back of the head




- standard

Hook fastener back of the head




Individual extra equipment


Openings

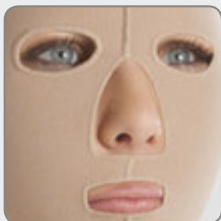

- eyes, nose, mouth and ears on request

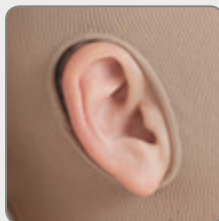

Knitted-in nose

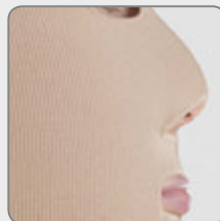

- compression in the nose area

Accessories

Lymphpad Line



- stimulates microcirculation in the tissue

Lymphpad Square



- stimulates microcirculation in the tissue

ScarPad light



- silicone patch for scars

Compression stump shrinkers (arm and leg)



MODELS





- Below-knee stump shrinker
- Stump shrinker after Pirogoff-amputation
- Above-knee stump shrinker
- Stump shrinker arm

EXTRA OPTIONS

1 Border / fastening leg

Top border	Silicone border (3.5 cm wide)	Silicone border (5 cm wide)	Hip attachment (single)	Sewn-on body part
				
<ul style="list-style-type: none"> • additional fastening required (e.g. adhesive lotion) 	<ul style="list-style-type: none"> • possible for below-knee stump shrinkers • silicone beads ensure high level of stability and breathability 	<ul style="list-style-type: none"> • possible for below-knee and above-knee stump shrinkers • silicone beads ensure high level of stability and breathability 	<ul style="list-style-type: none"> • optional additional silicone border for a better hold • can be worn on left or right 	<ul style="list-style-type: none"> • alternative fastening option with compression in the stomach area

Border / fastening arm

Top border	Silicone border (5 cm wide)	Bra fastening	Shoulder and strap
			
<ul style="list-style-type: none"> • additional fastening required (e.g. adhesive lotion) 	<ul style="list-style-type: none"> • 3.5 cm wide available on request • silicone beads ensure high level of stability and breathability 	<ul style="list-style-type: none"> • fastening option for attaching to the bra strap with velcro 	<ul style="list-style-type: none"> • alternative to silicone border

Individual extra equipment

Prosthesis pin opening

- 3 cm opening at the stump end

Accessories

Lymphpad Line



- stimulates microcirculation in the tissue

Lymphpad Square



- stimulates microcirculation in the tissue

ScarPad extra



- silicone pad for severely raised scars

ScarPad strong



- silicone pad for severely raised and extensive scarring

Special Detergent



- for medical compression garments and elastic fabrics
- highly concentrated and economical to use

Arion Proth-Pro



- donning aid for above-knee prosthesis with suction socket

Special Gloves



- rubberised palm
- makes donning of compression garments easier







Juzo SoftCompress and Juzo Lymphpads

The perfect supplement

Juzo SoftCompress

SoftCompress Breast Pad	SoftCompress Genital Pad Women	SoftCompress Knee Joint Pad and Elbow Joint Pad	SoftCompress Malleo Pad	SoftCompress Hand Pad
				
Art. no. 6837	Art. no. 6839	Art. no. 6845, 6842	Art. no. 6843	Art. no. 6844
<ul style="list-style-type: none"> to optimise pressure transfer in the compression bra or thorax garment, e.g. in the case of chest or thoracic wall oedema 	<ul style="list-style-type: none"> for insertion into a compression pantyhose targeted optimisation in pressure when treating genital oedemas as well as ideal pressure distribution 	<ul style="list-style-type: none"> to improve the compression in the hollow of the knee and in the crook of the arm 	<ul style="list-style-type: none"> to optimise the compression in the Bisgaard's region of the ankle targeted improvement of compression transfer and ideal pressure distribution 	<ul style="list-style-type: none"> especially for the back of the hand with two finger openings targeted improvement of compression transfer and ideal pressure distribution
<p>For further information please see the brochure "Juzo SoftCompress – Special products for oedema therapy".</p>				

Juzo Lymphpads and additional products

Lymphpad Line	Lymphpad Square	Pressure pad by Dr. Rotter	Malleolar pad made of silicone	Pressure pad for the hand	Liner Stocking Silver
					
Art. no. 9430	Art. no. 9431	Art. no. 9400, 9401 (antibacterial)	Art. no. 9408	long model (see picture): Art. no. 9420L, 9421L (antibacterial) short model: Art. no. 9420, 9421 (antibacterial)	Art. no. 9640
<ul style="list-style-type: none"> effective treatment aid for lymph- and lipoedema synthetic latex, polyamide fleece on the rear 	<ul style="list-style-type: none"> effective treatment aid for lymph- and lipoedema synthetic latex, polyamide fleece on the rear 	<ul style="list-style-type: none"> covered with textile or silver-containing (antibacterial) material 	<ul style="list-style-type: none"> improvement of compression in the Bisgaard's region soft and flat also available with textile elastic cover / cover containing silver 	<ul style="list-style-type: none"> covered with textile or silver-containing (antibacterial) material to be inserted between back of the hand and glove 	<ul style="list-style-type: none"> antibacterial thanks to the knitted-in silver thread soft fabric comfortable to wear for patients with extremely sensitive skin skin protection during donning and doffing securing of wound dressings or pressure pads
Size 20 x 29.5 cm (can be cut to any size)	Size 20 x 29.5 cm (can be cut to any size)				Size S / M: circumference "cC" < 50 cm Size L / XL: circumference "cC" > 50 cm

Very easy donning and doffing Juzo Arion Donning and Doffing Aids

TIP!
Visit www.juzo.com/donning for videos and instructions about the donning and doffing aids

Juzo Arion Easy-Slide Arm



Donning aid for compression sleeves

- sizes XS – L

Product benefits

- smooth, two-layer material
- for children in size XS (length 25.5 cm) also suitable as a donning and doffing aid for legs

Art. no. 1701

Juzo Arion Dactyna



Donning aid for compression gloves with **open** fingers

- sizes XS – L

Product benefits

- smooth, two-layer material

Art. Nr. 1708

Juzo Arion Easy-Slide Leg



Donning aid for compression stockings / pantyhose with **open** toes

- sizes S – XL

Product benefits

- smooth, two-layer material

Art. no. 1702

Juzo Arion Magnide



Donning aid for compression stockings / pantyhose with **closed** toes

- sizes M – XL

Product benefits

- smooth material
- with innovative magnetic closure

Art. no. 1703

Juzo Arion SlideX



Can only be used in combination with Juzo Arion Easy-Slide Leg or Juzo Arion Magnide

- sizes M – XL

Product benefits

- smooth material
- especially suitable for large leg circumferences
- extension for Juzo Arion Easy-Slide Leg and Juzo Arion Magnide

Art. no. 1704

Juzo Arion Proth-Pro



Donning aid for thigh prosthesis with suction socket

- sizes XS – XXL

Product benefits

- smooth material
- reduces the physical exertion required by the patient

Art. no. 1706

Juzo Arion Sim-Slide



Donning and doffing aid for compression stockings / pantyhose with **open** toes

- sizes M – XL

Product benefits

- smooth, two-layer material
- simply fasten to the ankle using the elastic strap

Art. no. 1705

Juzo Arion Easy-Slide Leg + Easy-Off



Donning and doffing aid for compression stockings / pantyhose with **open** toes

- sizes S – XL

Product benefits

- smooth, two-layer material
- doffing aid is also suitable for closed toes
- with velcro fastener for easy fixation above the ankle

Art. no. 1707

Practical tips for the supplier

Check the fit regularly

Compression garments need to fit perfectly. The accuracy of fit of the compression garment can diminish over time as e.g. the weight or size (e.g. reduction in the oedema) of the patient changes. The supplier should therefore check the fit of the garment on a regular basis and, if necessary, arrange for alterations to be made or recommend the purchase of a new garment.

Remind the patient of the wearing time

The compression garment should be worn for the period prescribed by the physician. This is the only way to guarantee an optimal therapy result. It is essential that the garment is worn during the day or as medically prescribed. For hygienic reasons and to allow the material to recover, the distributor should recommend to the patient that they change the compression garment on a daily basis. The patient should therefore ideally receive several pairs of the compression garment.

Explain how to care for the product

The compression garment should be washed daily at 40 °C to allow the material to regenerate. We recommend washing the garment with Juzo Special Detergent and to not use a fabric softener.

Instruct the patient on daily skin care

Dry skin can be kept supple by applying a cream and massaging it (e.g. with the Juzo Lymph Lotion) every day. To avoid inflammation, the skin should be kept clean and elastic at all times.


Warn against too much solar radiation

The patient should avoid direct exposure to sunlight at all times as this can promote the formation of skin changes and pigmentation disorders. The CCL 1 and 2 (18 – 32 mmHg) garments in Juzo's Expert product line offer a guaranteed and certified UV protection factor of 80.


Compression treatment with Juzo Expert and Juzo Expert Strong

A case study


Ursula K. (born 1954) contracted cervical cancer at the age of 39. After a Wertheim operation and removal of more than 30 lymph nodes and a lymphatic cyst in the lower abdomen, Ursula K. immediately developed lymphoedema in both legs. She has been undergoing MLD and compression treatment since 1993 on account of the lymphoedema. Initially she was provided with circular knitted garments, but with less than optimum results. Ursula K. discovered Juzo Expert by chance approx. 3 years ago. She has been wearing Juzo Expert and Juzo Expert Strong garments since then.

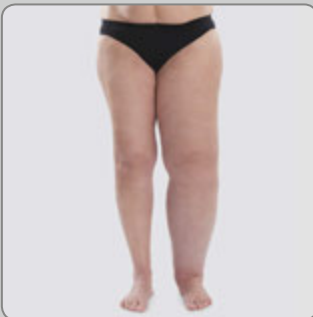

Secondary lymphoedema in both legs after multiple operations following a diagnosis of cervical cancer

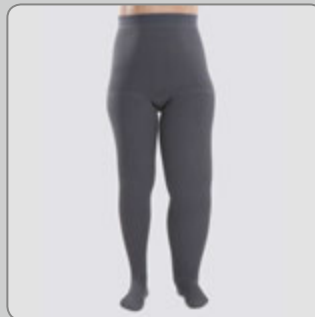

Treatment example: pantyhose (AT), Juzo Expert

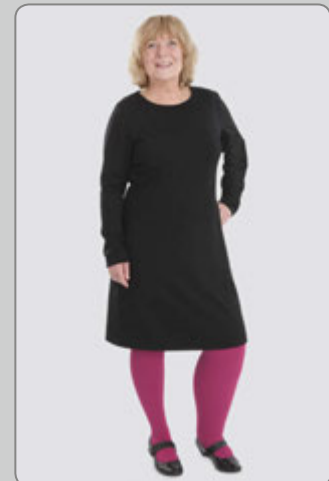

Today Ursula K. wears stockings (AG) (left leg: Juzo Expert Strong, CCL 3; right leg: Juzo Expert, CCL 2) underneath leggings (Juzo Expert, CCL 2)

International
Julius Zorn GmbH
Juliusplatz 1
86551 Aichach
Germany
info@juzo.com

www.juzo.com

UK
Juzo UK Ltd
South Court
Sharston Rd
Sharston
Manchester M22 4SN
Great Britain
info@juzo.co.uk

www.juzo.co.uk

