


## Special-purpose garments

Customised compression products for lymphology – for maximum therapeutic success with more comfort

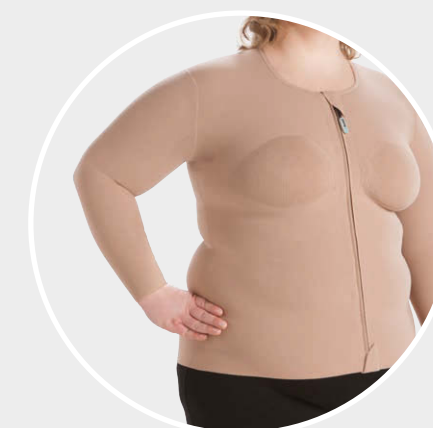

### Juzo Customer Service

**Europe** · Phone: +49 (0)8251 901 300 · +49 (0)8251 901 321 · E-mail: [order@juzo.com](mailto:order@juzo.com)  
**UK** · Phone: +44 (0) 161 358 0104 · Fax: +44 (0) 161 945 2786 · E-mail: [sales@juzo.co.uk](mailto:sales@juzo.co.uk)

**Julius Zorn GmbH** · Juliusplatz 1 · 86551 Aichach · Germany · [www.juzo.com](http://www.juzo.com)

**Juzo UK Ltd** · South Court · Sharston Rd · Sharston · Manchester M22 4SN · Great Britain · [info@juzo.co.uk](mailto:info@juzo.co.uk) · [www.juzo.co.uk](http://www.juzo.co.uk)

ENG · 14290003/8/8 · 8408 LY · 03/2019  
Subject to modifications and errors.



[www.juzo.com](http://www.juzo.com)

Freedom in Motion

# Customer Service department

## Juzo – Always there for you

Engaging, friendly and competent – that is how our customers describe us. We look after our customers with a smile and genuine interest in their wishes. Our goal is to answer their needs as best as possible, which is why we are always on the lookout for new, optimised solutions. In these efforts, we rely on our experience and our unique expertise, which we continue to develop further every day. We are always there for you with help and advice – not just to fulfil our own claims – but to assist you in supporting the therapy success of your patients with the right products. We are looking forward to your call and finding a solution that's right for you – because “Can't be done” is never an option for us!

### Our Customer Service department offers:

- + general information
- + individual product advice
- + order acceptance
- + feedback on the status of your order
- + acceptance of alteration or repair orders
- + advice on questions about compression therapy

### Contact

You can contact us by phone **Monday to Friday**.  
Please feel free to send us an e-mail or a fax.

#### Juzo Customer Service International

Phone +49 (0)8251 901 300  
Fax +49 (0)8251 901 321  
E-mail [order@juzo.com](mailto:order@juzo.com)

#### Office hours

7:00 am – 6:00 pm (CET)

#### Juzo Customer Service UK

Phone +44 (0)161 358 0104  
Fax +44 (0)161 945 2786  
E-mail [sales@juzo.co.uk](mailto:sales@juzo.co.uk)

#### Office hours

9:00 am – 5:00 pm (CET)

# Here's how to find the right product quickly

## General information

Effective oedema therapy	04
Special products for decongestion	05
Lymphology expert	06
Practical tips for caregivers	47
Case study	47

## Product overview

Toe-portions	09
Below-knee stockings · AD	11
Thigh-high stockings · AG	15
Pantyhose · AT	19
Gloves	27
Sleeves	31
Thorax garments	35
Head garments	39
Stump garments (arm and leg)	43
Additional products	45

# Effective oedema therapy


## The components

### Components of oedema therapy

- + manual lymphatic drainage (MLD)
- + compression therapy
- + decongesting movement and breath therapy
- + skin care and skin rehabilitation
- + education and training for individual self-therapy

### Goals of oedema therapy

- + improvement of lymph flow
- + loosening of hardened, fibrotic tissue
- + reduction of connective tissue increase
- + improvement of mobility in the affected limbs in order to increase effectiveness of the muscle and joint pump
- + for lipoedema: pain alleviation, reduction of haematoma and oedema

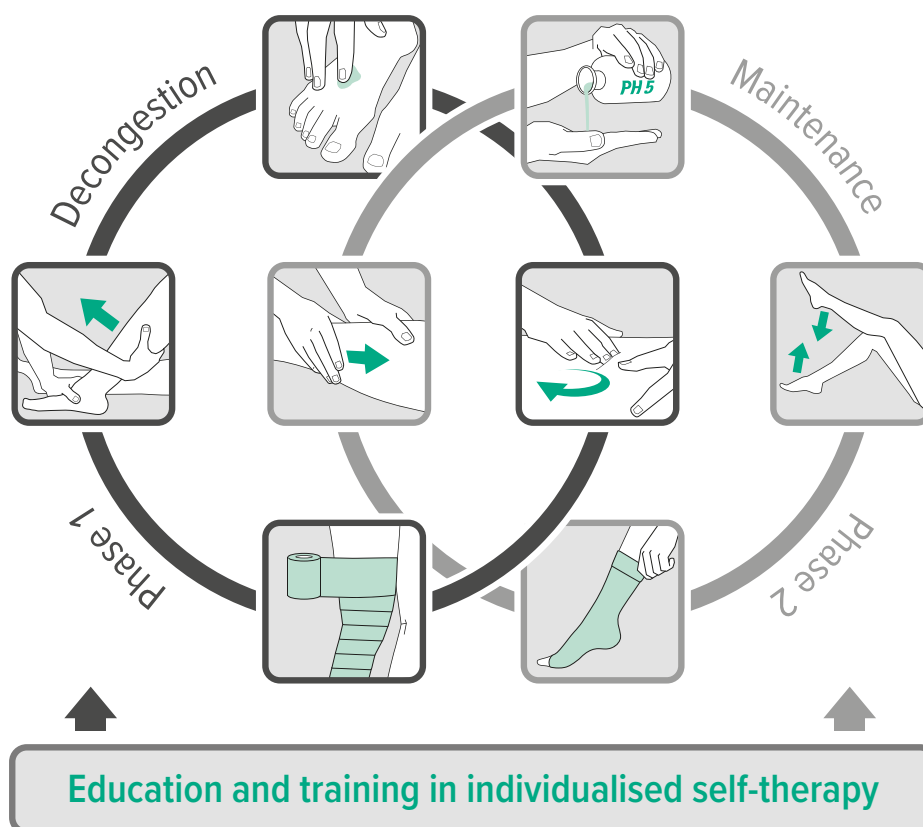

### Phase 1: Decongestion phase

Using manual lymphatic drainage (MLD), the transport capacity of the lymphatic system is increased in order to enable mobilisation and removal of protein-rich oedema fluid (lymph). The decongested limb is then compressed with compression bandages or supports (e.g. Juzo SoftCompress). This prevents the return flow of lymphatic fluid and reduces the increase in connective tissue.

### Phase 2: Maintenance phase

In the second phase, the applications of MLD are reduced and compression bandages or supports are replaced by flat-knit compression garments (e.g. Juzo Expert / Juzo Expert Strong). This optimises and maintains long-term treatment success.

In both phases, consistent care protects the skin from drying out and minimises the risk of infection. Physical activities and decongestive exercises also help to stimulate lymphangiomotoricity. The active engagement of the patient is important for the long-term success of the treatment.




# Special products for decongestion

## Juzo SoftCompress and Juzo ACS Light


### Compression bandages

**Juzo SoftCompress** can significantly increase the success of decongestive therapy. Bandaging aids and additional products feature a particularly skin-friendly cotton surface, as opposed to frequently used synthetic materials. This provides the patient with pleasant wearing comfort. Vertical foam channels form a unique surface structure that supports lymphatic drainage. In addition, Juzo SoftCompress bandaging aids make it easy for patients to use the bandages, therefore ensuring the long-term success of the treatment.

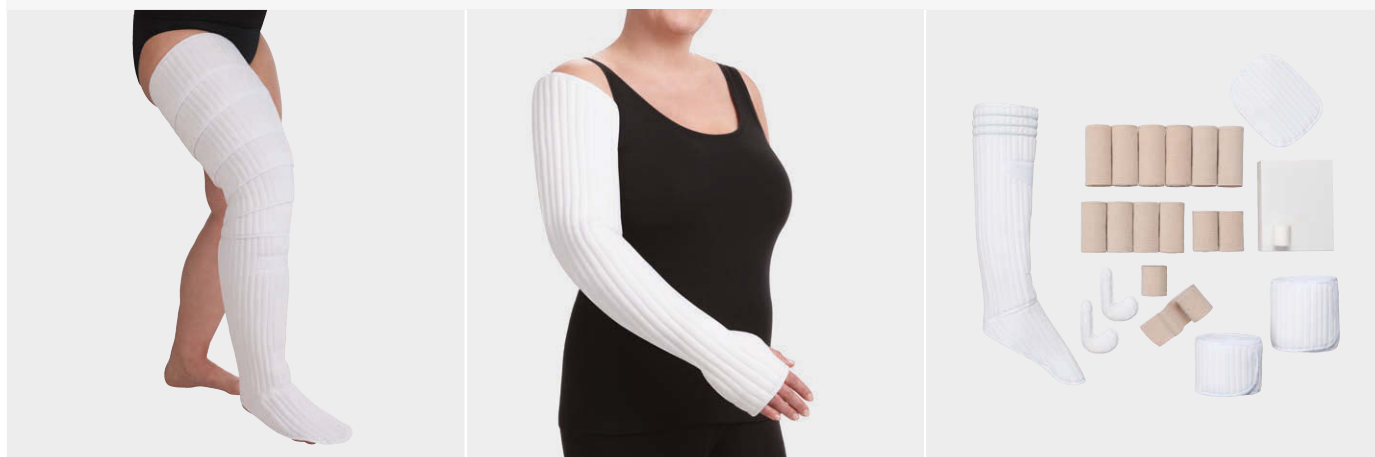

### Adjustable compression systems

**Juzo ACS Light** short-stretch compression segments are used for additional treatment during decongestive therapy. The products are easy to put on, thanks to the incorporated lining (fixing aid). They can be adapted to the individual patient using adjustable fasteners – even if circumferences change during the therapy. This uncomplicated application increases patient compliance and supports self-therapy. With Juzo ACS Light, therapeutic compression up to 40 mmHg can be achieved.

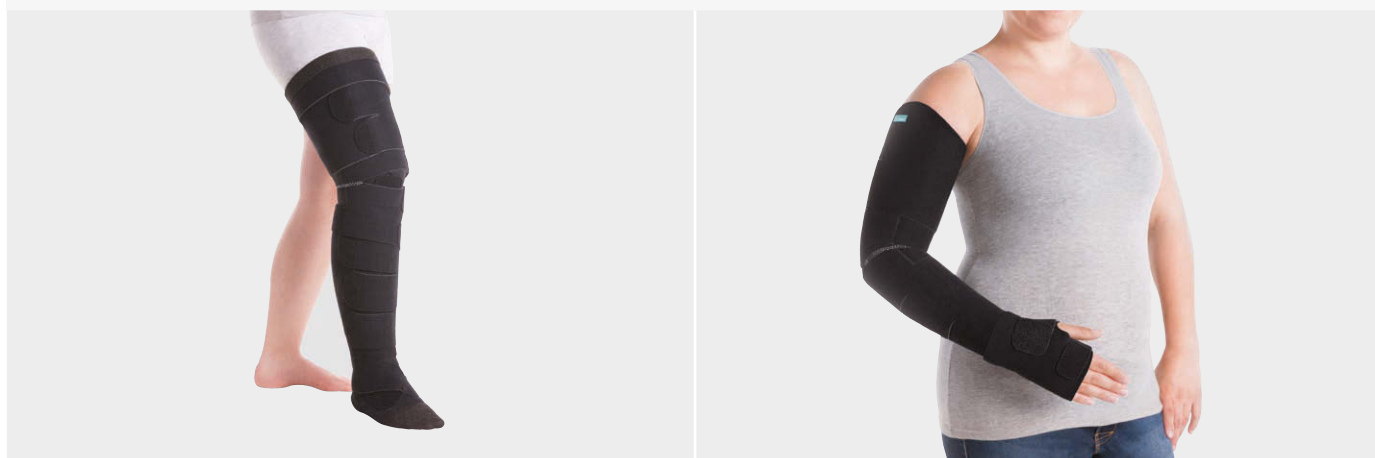

Juzo SoftCompress and Juzo ACS Light products can also be used together.  
Further information is available in the brochure Juzo special products for decongestion.



# Lymphology expert

## Versatile compression products from head to toe

### Lymphology applications

Lymphology treatment approaches mainly utilise flat-knitted products with seams, which, along with lymphatic drainage, form the basis for all types of oedema therapies. Compression garments ensure the long-term success of decongestive therapies and are generally indispensable in lymphology. From the foot-toe-portion to stockings, pantyhose and face masks – treatment covers all areas of the body. The flat-knitted products are precisely tailored to the patient's body measurements and, depending on the area of application, are also available in different versions. The combination of various materials here – e.g. with cotton or silver – makes it possible for Juzo to offer products for diverse patient requirements.

### How they work

The strong knitted fabrics stand through special features that include a massage effect that promotes lymph drainage. The fabrics also do not yield to the oedema. During movement, a high working pressure is generated that enables optimal compression of the tissue affected by the oedema, which is ideal for a lymphology treatment approach.


### Convincing product benefits

- + optimal wearing comfort for patients due to the particularly flat and elastic seams
- + the homogeneous, soft and smooth material ensures patient mobility is maintained
- + exceptionally skin-friendly due to the double-layered textile-yarn winding in the compression threads
- + compression garments can be worn normally every day, thanks to the use of latex-free materials that are more resistant to sunlight and ointments and are more resilient, yet more breathable as well
- + easy to put on and take off, thanks to the knitted fabric's fine mesh surface structure and optional integration of additional fasteners (hook fastener, zipper, velcro fastener)


## Juzo Expert

For individual oedema treatment – also with silver thread or cotton


Juzo Expert offers maximum freedom of movement and is particularly pleasant to wear. The knitted fabric is perfect for flat-knit beginners and whole body treatments. The knitted fabric used in Juzo Expert Silver not only offers compression but also has a natural antibacterial effect due to the knitted-in silver threads. Juzo Expert Silver restores the balance of inflamed or irritated skin and thus reduces itching. The use of natural fibres makes Juzo Expert Cotton extra soft. This stocking is made of nearly 50 % cotton; the skin is covered with a pure cotton fabric. Natural fibres make the stocking soft and easy to put on and take off, which means Juzo Expert Cotton is particularly comfortable for delicate and sensitive skin.

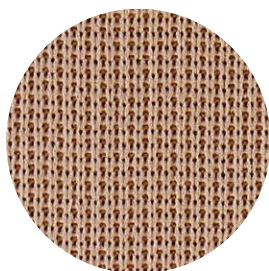

Juzo Expert

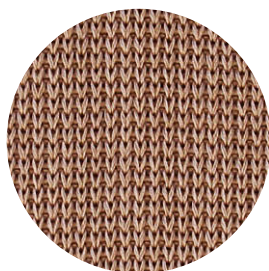

Juzo Expert Silver

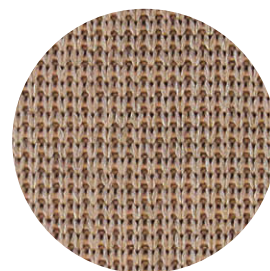


Juzo Expert Cotton


## Juzo Expert Strong

Strong for pronounced oedema – also with silver thread or cotton


Juzo Expert Strong is characterised by its high working pressure and activating massage effect that stimulates lymphatic drainage. The firm and strong fabric provides effective support in the treatment of pronounced lymphoedemas and lipoedemas. Juzo Expert Strong Silver provides individual solutions for the most difficult lymphological cases. The strong knitted fabric with an additional knitted-in silver thread generates high working pressure and also has an antibacterial effect that combats bacteria on the skin, such as staphylococcus aureus. Juzo Expert Strong Cotton combines the reliability and strength of Juzo Expert Strong with natural cotton fibres that ensure exceptional wearing comfort. The product is perfect for patients with sensitive skin and anyone who likes to wear cotton.

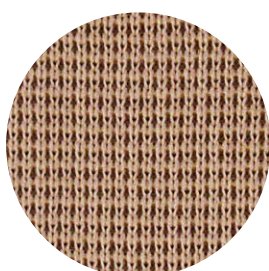

Juzo Expert Strong

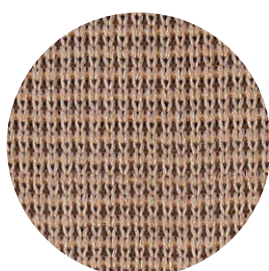

Juzo Expert Strong Silver

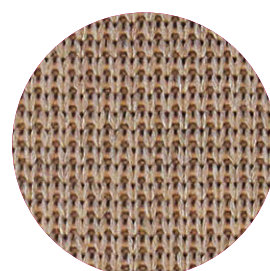

Juzo Expert Strong Cotton

## Colours

Available for Juzo Expert and Juzo Expert Strong (CCL 1 – 3)\*

	Sugar		Cacao		Black pepper
	Cardamom		Poppy seed		Trend Colours
	Almond		Blueberry		Batik Collection

For current Trend Colours and the Batik Collection, see the Juzo lymphology product overview or visit: [www.juzo.com/trend-colours](http://www.juzo.com/trend-colours)  
The Batik Collection is available in all colours for Juzo Expert (except Sugar and Almond) in a black or white batik version as well.

\* CCL 4, models in Silver and Cotton available in Almond





Freedom in Motion





Toe-portions

# Toe-portions

- Available in
- Juzo Expert
  - Juzo Expert Silver
  - Juzo Expert Cotton
  - Juzo Expert Strong
  - Juzo Expert Strong Silver





Below-knee stockings · AD

# Below-knee stockings · AD

- Available in
- Juzo Expert
  - Juzo Expert Silver
  - Juzo Expert Cotton
  - Juzo Expert Strong
  - Juzo Expert Strong Silver
  - Juzo Expert Strong Cotton

AD



Model variants



AD




AB




BD

Toes




Ball stub with open toes

- For oedema on the top of the foot
- Distal diagonal border




Ball stub with closed toes

- For oedema on the top of the foot
- Distal diagonal border




Open toes



Closed toes







- Very elastic
- High degree of wearing comfort












Sewn-on foot-toe-portion

- No overlapping of stocking and foot-toe-portion
- Little toe (5) can be open



Border / Fastening			
			
<b>Overheight / max. overheight</b> <ul style="list-style-type: none"> <li>· Lateral diagonal border</li> <li>· Optimised pressure discharge to proximal</li> </ul>	<b>Top border</b> <ul style="list-style-type: none"> <li>· Additional fastening option available if needed (e.g. adhesive lotion)</li> </ul>	<b>Silicone border · 5 cm</b> <b>Silicone border pattern · 5 cm</b> <ul style="list-style-type: none"> <li>· Silicone border: 3.5 cm also possible</li> <li>· Silicone nubs ensure a high degree of stability and breathability</li> </ul>	<b>Balance silicone border · 5 cm</b> <b>Balance silicone border pattern · 5 cm</b> <ul style="list-style-type: none"> <li>· Balance silicone border: 3.5 cm also possible</li> <li>· Especially high breathability</li> <li>· Low shear forces</li> </ul>
			
<b>Lace silicone border · 5 cm</b> <ul style="list-style-type: none"> <li>· Circumferential silicone strips ensure particularly good adhesion</li> </ul>	<b>Elastic border · 3.5 cm</b> <ul style="list-style-type: none"> <li>· Stable border</li> <li>· Silicone-free</li> </ul>		

Extra options			
			
<b>Additional instep stitching</b>	<b>Silicone stopper</b>	<b>Tricot lining at “cY”</b>	<b>Integrated pressure pad (Dr. Rotter)</b>
<ul style="list-style-type: none"><li>· Y-stitching /90° foot</li><li>· Less creasing in the stretched/instep area, thanks to the anatomically shaped heel</li></ul>	<ul style="list-style-type: none"><li>· Possible positioning: lateral at the outer side, across</li></ul>	<ul style="list-style-type: none"><li>· For sensitive skin in the stretched/instep area</li><li>· Also possible with silver in case of skin irritation</li></ul>	<ul style="list-style-type: none"><li>· Improvement of compression in the retromalleolar area</li><li>· Also possible with silver in case of skin irritation</li><li>· Malleoli pads made of silicone also available as an alternative</li></ul>
			
<b>Sewn-in silicone border on the inside</b>	<b>Incorporated pockets for pads</b>	<b>Seams on the outside</b>	<b>Zipper</b>
<ul style="list-style-type: none"><li>· Only possible with overheight</li><li>· 3/4 sewn-in on the inside also possible</li><li>· Can be combined with silicone border at the top</li></ul>	<ul style="list-style-type: none"><li>· Customisation based on markings</li><li>· Also possible with silver in case of skin irritation</li></ul>	<ul style="list-style-type: none"><li>· Particularly flat seam touches the skin</li><li>· Suitable for sensitive skin</li></ul>	<ul style="list-style-type: none"><li>· Standard in the seam</li><li>· Individual positioning is possible</li></ul>
			
<b>Silicone coating on the sole of the foot</b>			
<ul style="list-style-type: none"><li>· Anti-slip coating</li></ul>			



# Thigh-high stockings · AG

- Available in
- Juzo Expert
  - Juzo Expert Silver
  - Juzo Expert Cotton
  - Juzo Expert Strong
  - Juzo Expert Strong Silver
  - Juzo Expert Strong Cotton



Toes

Ball stub with open toes

- For oedema on the top of the foot
- Distal diagonal border

Ball stub with closed toes

- For oedema on the top of the foot
- Distal diagonal border

Open toes

Closed toes

- Very elastic
- High degree of wearing comfort

Sewn-on foot-toe-portion

- No overlapping of stocking and foot-toe-portion
- Little toe (5) can be open



## Border/ Fastening

			
<b>Side overheight / max. overheight</b>	<b>Overheight at the front of the leg</b>	<b>Top border</b>	<b>Silicone border · 5 cm</b> <b>Silicone border pattern · 5 cm</b>
<ul style="list-style-type: none"> <li>· Lateral diagonal border</li> <li>· Optimised pressure discharge to proximal</li> </ul>	<ul style="list-style-type: none"> <li>· Raised for a better fit for patients with ventrolateral thigh oedema</li> </ul>	<ul style="list-style-type: none"> <li>· Additional fastening options available if needed (e.g. adhesive lotion)</li> </ul>	<ul style="list-style-type: none"> <li>· Silicone nubs ensure a high degree of stability and breathability</li> </ul>
			
<b>Balance silicone border · 5 cm</b> <b>Balance silicone border pattern · 5 cm</b>	<b>Lace silicone border · 5 cm</b>	<b>Hip attachment · (individual)</b>	<b>Hip attachment · (pair)</b>
<ul style="list-style-type: none"> <li>· Especially high breathability</li> <li>· Low shear forces</li> </ul>	<ul style="list-style-type: none"> <li>· Circumferential silicone strips ensure particularly good adhesion</li> </ul>	<ul style="list-style-type: none"> <li>· Additional silicone border for better fixation is possible</li> <li>· Worn on the left or right</li> </ul>	<ul style="list-style-type: none"> <li>· Opened at the front and back</li> <li>· Available with a strap on request (opened at the front)</li> <li>· Additional silicone border is possible</li> </ul>
			
<b>Sewn-on body part</b>			
<ul style="list-style-type: none"> <li>· Alternative fastening option with compression in the stomach area</li> </ul>			

## Extra options

**Additional instep stitching**

- Y-stitching /90° foot
- Less creasing in the stretched/instep area, thanks to the anatomically shaped heel

**Anatomically flexed form at "cE"**

- 30° or 50° angulation possible
- In the case of a predominately angled knee position

**Silicone stopper**

- Possible positioning on the thigh: front on the side, lengthwise; across on the outside

**Sewn-in silicone border on the inside**

- Only possible with overheight
- 3/4 sewn-in on the inside also possible
- Can be combined with silicone border at the top

**Tricot lining at "cY"**

- For sensitive skin in the stretched/instep area
- Also possible with silver in case of skin irritation

**Tricot lining at "cE"**

- For sensitive skin at the back of the knee
- Also possible with silver in case of skin irritation

**Integrated pressure pad (Dr. Rotter)**

- Improvement of compression in the retromalleolar area
- Also possible with silver in case of skin irritation
- Malleoli pads made of silicone available

**Incorporated pockets for pads**

- Customisation based on markings
- Also possible with silver in case of skin irritation

**Seams on the outside**

- Particularly flat seam touches the skin
- Suitable for sensitive skin

**Porosa starting at "cD"**

- Compression reduction starting at "cD"

**Zipper**

- Standard in the seam
- Individual positioning is possible

**Bilateral dual spiral stays and/or patella ring**

- Integrated knee support

**Silicone coating on the sole of the foot**



- Anti-slip coating





# Pantyhose · AT

- Available in
- Juzo Expert
  - Juzo Expert Silver
  - Juzo Expert Cotton
  - Juzo Expert Strong
  - Juzo Expert Strong Silver
  - Juzo Expert Strong Cotton

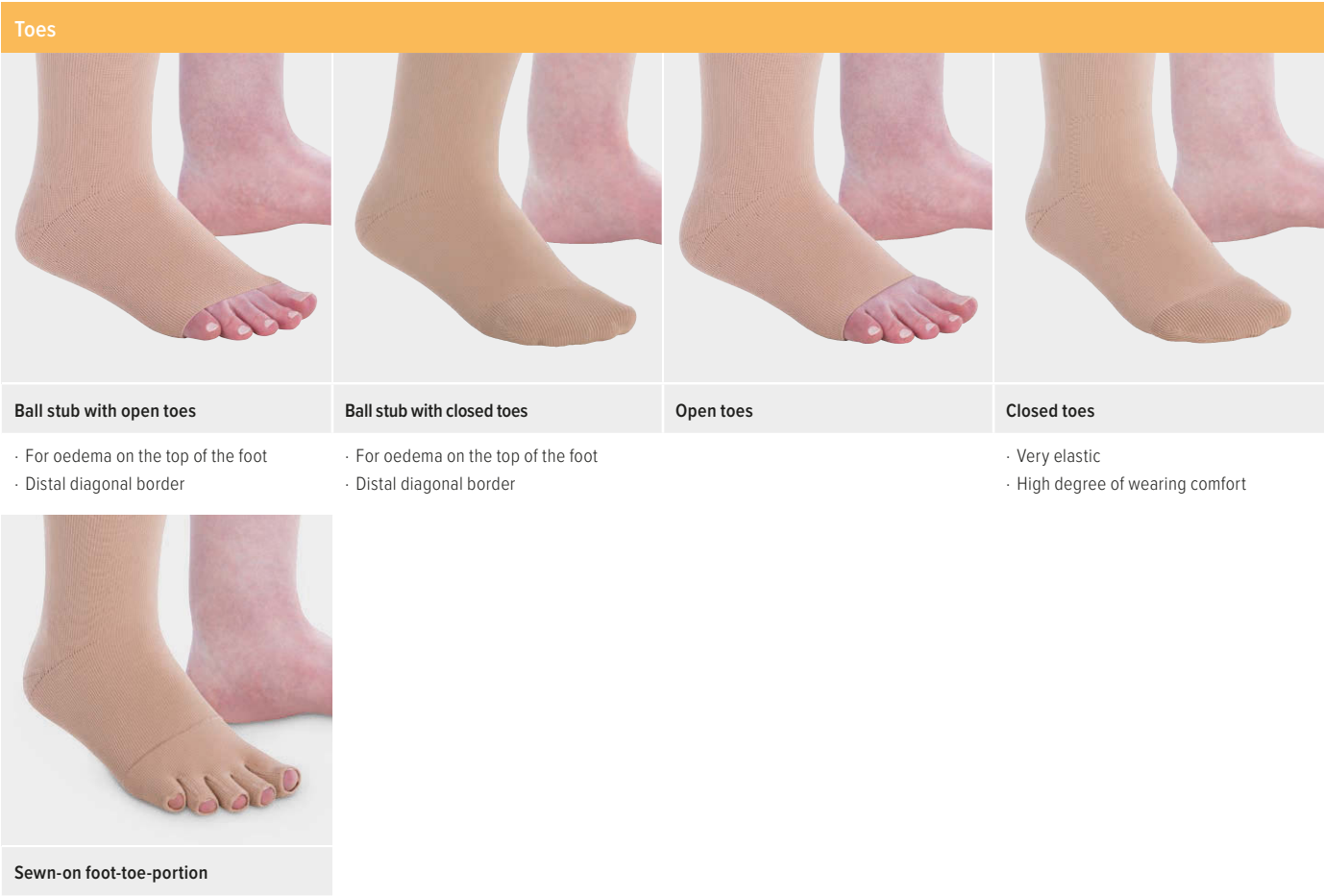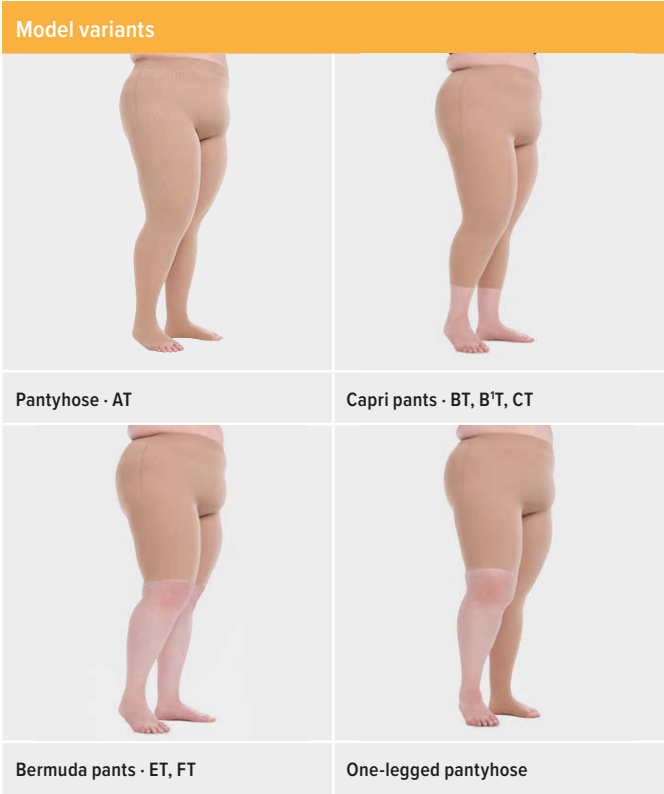
















## Border/ Fastening

			
<b>Oblique version</b> <ul style="list-style-type: none"> <li>Standard</li> <li>In case of frontal oedema development</li> <li>In case of a flat inguinal orientation</li> </ul>	<b>Box type</b> <ul style="list-style-type: none"> <li>For large circumference dimensions</li> <li>In case of lateral oedema development</li> <li>In case of pronounced hip and buttock shapes</li> </ul>	<b>Slip form</b> <ul style="list-style-type: none"> <li>In case of less pronounced stages of oedema and body shapes</li> <li>In case of a steep inguinal orientation</li> <li>In case of a flat abdominal area</li> <li>Abdominal circumference: max. 120 cm</li> </ul>	<b>Mat fit (pregnancy fastening)</b> <ul style="list-style-type: none"> <li>Body part in the back area CCL 1</li> <li>Body part without compression (grows with the body)</li> <li>Velcro fastener can be on the right or left</li> <li>Minimum body part length: 34 cm</li> </ul>
			
<b>Top border</b> <ul style="list-style-type: none"> <li>Soft border</li> </ul>	<b>Adjustable waistband</b> <ul style="list-style-type: none"> <li>Retractable border enables individual adjustment options</li> </ul>	<b>Waist band</b> <ul style="list-style-type: none"> <li>Flat and elastic border</li> <li>The border of the knitted fabric will not fold over</li> </ul>	<b>Waist band with velcro fastener</b> <ul style="list-style-type: none"> <li>Flat and elastic border enables adjustment</li> </ul>
			
<b>Silicone border - 5 cm</b> <ul style="list-style-type: none"> <li>Flat border</li> <li>Additional fastening with silicone nubs</li> </ul>	<b>Body part with zipper</b> <ul style="list-style-type: none"> <li>In the middle as standard</li> <li>Individual positioning is possible</li> </ul>	<b>Body part with hook fastener</b> <ul style="list-style-type: none"> <li>In the middle as standard</li> <li>Individual positioning is possible</li> </ul>	<b>Body part with velcro fastener</b> <ul style="list-style-type: none"> <li>In the middle as standard</li> <li>Individual positioning is possible</li> </ul>









## Standard combinations (all other combinations are possible)

## AT body parts in general





	CCL			
AT	1	2	3	4
Leg parts	1	2	3	4
Body part	1	2	2	2

Extra options			
			
<b>Gusset</b>	<b>Small gusset</b>	<b>Mesh gusset</b>	<b>Small mesh gusset</b>
<ul style="list-style-type: none"> <li>· Knitted</li> <li>· Compressive gusset also possible</li> </ul>	<ul style="list-style-type: none"> <li>· Beneficial in the case of large thigh circumferences</li> <li>· Compressive gusset also possible</li> </ul>	<ul style="list-style-type: none"> <li>· Better air circulation</li> </ul>	<ul style="list-style-type: none"> <li>· Beneficial in the case of large thigh circumferences</li> <li>· Better air circulation</li> </ul>
			
<b>Fly</b>	<b>Scrotum</b>	<b>Stoma opening</b>	<b>Open crotch</b>
<ul style="list-style-type: none"> <li>· For men</li> <li>· Horizontal design</li> </ul>	<ul style="list-style-type: none"> <li>· Optimum fit in the genital area for men</li> <li>· Also available in combination with a fly</li> <li>· Mesh material also possible</li> </ul>	<ul style="list-style-type: none"> <li>· Customisation based on markings</li> </ul>	<ul style="list-style-type: none"> <li>· Without gusset</li> <li>· Not recommended in the case of adipose tissue</li> </ul>
			
<b>Additional instep stitching</b>	<b>Anatomically flexed form at "cE"</b>	<b>Adhesive borders</b>	
<ul style="list-style-type: none"> <li>· Less creasing in the stretched/instep area, thanks to the anatomically shaped heel</li> </ul>	<ul style="list-style-type: none"> <li>· 30° or 50° angulation possible</li> <li>· In the case of a predominately angled knee position</li> </ul>	<ul style="list-style-type: none"> <li>· Placement options on thigh: lateral at the outer side, lengthwise at the front</li> </ul>	
			
<b>Sewn-in 3/4 silicone border on the inside</b>			
<ul style="list-style-type: none"> <li>· On the thigh</li> <li>· Increased stability and adhesion</li> <li>· Only possible for body part in the oblique version</li> </ul>			





## Extra options

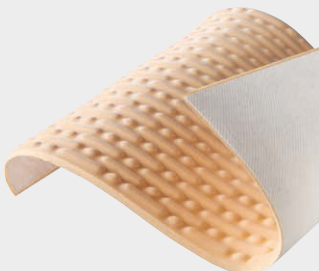			
<b>Tricot lining at "cY"</b>	<b>Tricot lining at "cE"</b>	<b>Integrated pressure pad (Dr. Rotter)</b>	<b>Incorporated pockets for pads</b>
<ul style="list-style-type: none"> <li>· For sensitive skin in the stretched/instep area</li> <li>· Also possible with silver in case of skin irritation</li> </ul>	<ul style="list-style-type: none"> <li>· For sensitive skin at the back of the knee</li> <li>· Also possible with silver in case of skin irritation</li> </ul>	<ul style="list-style-type: none"> <li>· Improvement of compression level in the retromalleolar area</li> <li>· Also possible with silver in case of skin irritation</li> <li>· Malleoli pads made of silicone also available as an alternative</li> </ul>	<ul style="list-style-type: none"> <li>· Customisation based on markings</li> <li>· Also possible with silver in case of skin irritation</li> </ul>
			
<b>Seams on the outside</b>	<b>Zipper</b>	<b>Bilateral dual spiral stays and/or patella ring</b>	<b>Silicone coating on the sole of the foot</b>
<ul style="list-style-type: none"> <li>· Particularly flat seam touches the skin</li> <li>· Suitable for sensitive skin</li> </ul>	<ul style="list-style-type: none"> <li>· Standard in the seam</li> <li>· Individual positioning is possible</li> </ul>	<ul style="list-style-type: none"> <li>· Integrated knee support</li> </ul>	<ul style="list-style-type: none"> <li>· Anti-slip coating</li> </ul>









## Accessories · AD, AG and AT

Pads		Lymphpads	
			
<b>Pressure pad (Dr. Rotter)</b>	<b>Malleoli pads</b>	<b>Lymphpad Line</b>	<b>Lymphpad Square</b>
<ul style="list-style-type: none"> <li>· For improving pressure in the ankle area</li> <li>· Made of carbosan moulded foam</li> <li>· With textile elastic cover or cover containing silver</li> </ul>	<ul style="list-style-type: none"> <li>· For improving pressure in the ankle area</li> <li>· Made of silicone; soft and flat</li> <li>· Uncovered or with textile elastic cover or cover containing silver</li> </ul>	<ul style="list-style-type: none"> <li>· Stimulates micro circulation</li> <li>· Synthetic latex, reverse side made of polyamide non-woven material</li> <li>· Size 20 x 29.5 cm</li> </ul>	<ul style="list-style-type: none"> <li>· Stimulates micro circulation</li> <li>· Synthetic latex, reverse side made of polyamide non-woven material</li> <li>· Size 20 x 29.5 cm</li> </ul>

SoftCompress			
			
<b>Malleo pad</b>	<b>Knee pad</b>	<b>Genital pad for women</b>	<b>Genital pad for men</b>
<ul style="list-style-type: none"> <li>· For improving pressure in the ankle area</li> <li>· With a skin-friendly cotton cover</li> </ul>	<ul style="list-style-type: none"> <li>· Optimisation of the compression ratio at the back of the knee</li> </ul>	<ul style="list-style-type: none"> <li>· For insertion into a compression pantyhose</li> <li>· Targeted pressure optimisation in the case of genital oedema; ideal pressure distribution</li> </ul>	<ul style="list-style-type: none"> <li>· For insertion into a compression pantyhose</li> <li>· Targeted transfer of pressure and padding for the male genital area</li> </ul>


Fastening	Care products		
			
<b>Adhesive lotion</b>	<b>Lymph lotion</b>	<b>Vital Balsam 7</b>	<b>Special Detergent</b>
<ul style="list-style-type: none"> <li>· Special roll-on for securely holding stockings and sleeves</li> </ul>	<ul style="list-style-type: none"> <li>· Moisturising lotion with 4% urea for targeted skincare in cases of lymphoedema and lipoedema</li> </ul>	<ul style="list-style-type: none"> <li>· Cooling and caring balm for smooth skin during compression therapy</li> </ul>	<ul style="list-style-type: none"> <li>· Particularly mild and economical concentrated detergent for gentle cleaning of compression garments</li> </ul>



Donning and doffing aids			
			
<b>EasyFit XL</b>	<b>Slippie Leg</b>	<b>Arion Easy-Slide Leg</b>	<b>Arion Sim-Slide®</b>
<ul style="list-style-type: none"> <li>· Donning and doffing aid</li> <li>· For open/closed toes</li> </ul>	<ul style="list-style-type: none"> <li>· Donning aid made of low friction Tyvek</li> <li>· For open toes</li> </ul>	<ul style="list-style-type: none"> <li>· Donning aid for open toes</li> </ul>	<ul style="list-style-type: none"> <li>· Donning and doffing aid for open toes</li> </ul>
Donning and doffing aids		Accessories	Liner stocking
			
<b>Arion Magnide®</b>	<b>Arion Easy-Slide Leg + Easy-Off</b>	<b>Special gloves</b>	<b>Liner Silver</b>
<ul style="list-style-type: none"> <li>· Donning aid for closed toes</li> </ul>	<ul style="list-style-type: none"> <li>· Set consisting of donning and doffing aid for open toes</li> </ul>	<ul style="list-style-type: none"> <li>· Anti-slip coating</li> <li>· Simplifies donning of compression garments</li> </ul>	<ul style="list-style-type: none"> <li>· With antibacterial silver thread</li> <li>· Fixation of wound dressings and pads</li> </ul>



Freedom in Motion

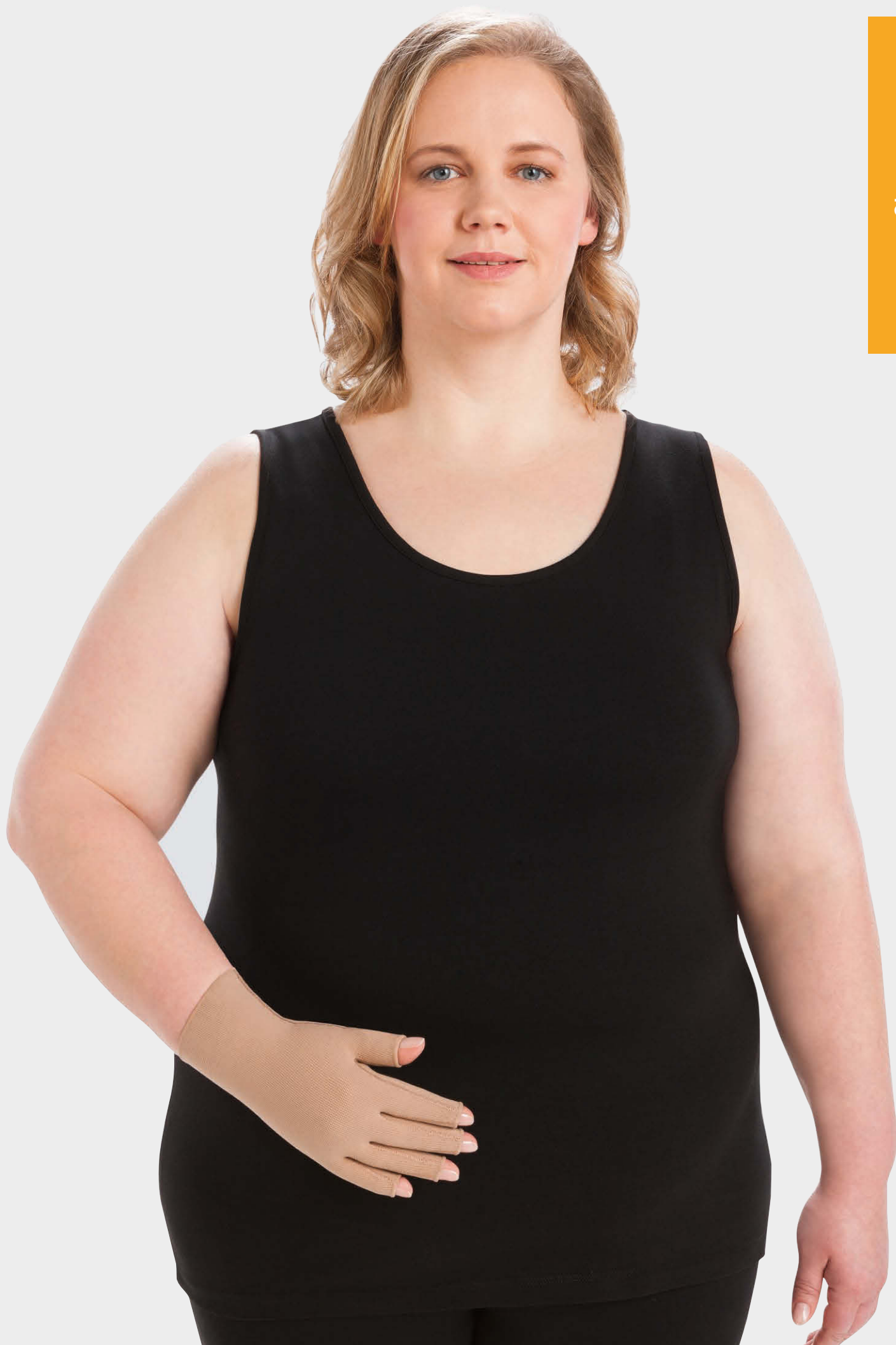







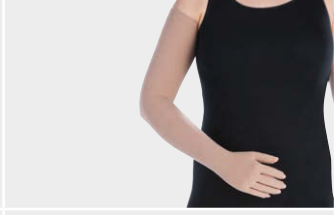
# Gloves

- Available in
- Juzo Expert
  - Juzo Expert Silver
  - Juzo Expert Cotton
- Juzo Expert Strong
  - Juzo Expert Strong Silver





Glove

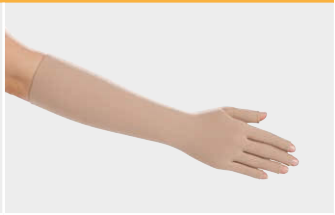

Model variants

	
Glove	Long glove
	
Finger support	One-piece sleeve and glove

Finger

			
Open fingers	Closed fingers	Hand portion with open thumb stub	Open finger support

· Thumb closed also possible

· Finger support closed also possible

Border

			
Overheight	Silicone border · 3.5 cm	Balance silicone border · 3,5 cm	Longer glove

· Lateral diagonal border  
· Optimised pressure discharge to proximal







· Silicone nubs ensure a high degree of stability and breathability

· Especially high breathability  
· Low shear forces










· Individual lengths possible up to max. "cE"

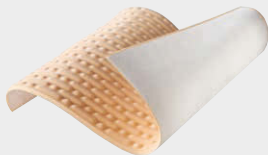

## Extra options

			
<b>Zipper</b>	<b>Incorporated pockets for pads</b>	<b>Silicone coating</b>	<b>Tricot lining</b>
<ul style="list-style-type: none"> <li>· Individual positioning</li> </ul>	<ul style="list-style-type: none"> <li>· Pocket for standard pad</li> <li>· Customisation based on markings</li> <li>· Also possible with silver in case of skin irritation</li> </ul>	<ul style="list-style-type: none"> <li>· Improved grip</li> <li>· Possible with open and closed fingers</li> </ul>	<ul style="list-style-type: none"> <li>· Between the thumb and index finger</li> <li>· For sensitive skin</li> <li>· Also possible with silver in case of skin irritation</li> </ul>
			
<b>Seams on the outside</b>	<b>Closed finger support with fastener</b>		
<ul style="list-style-type: none"> <li>· Particularly flat seam touches the skin</li> <li>· Suitable for sensitive skin</li> </ul>	<ul style="list-style-type: none"> <li>· With velcro fastener</li> <li>· Open finger support also possible</li> </ul>		

## Accessories · Gloves


Lymphpads		Pads	
			
<b>Lymphpad Line</b>	<b>Lymphpad Square</b>	<b>Pressure pad · Long</b>	<b>Pressure pad · Short</b>
<ul style="list-style-type: none"> <li>· Stimulates micro circulation</li> <li>· Synthetic latex, reverse side made of polyamide non-woven material</li> <li>· Size 20 x 29.5 cm</li> </ul>	<ul style="list-style-type: none"> <li>· Stimulates micro circulation</li> <li>· Synthetic latex, reverse side made of polyamide non-woven material</li> <li>· Size 20 x 29.5 cm</li> </ul>	<ul style="list-style-type: none"> <li>· Improved transfer of pressure to the back of the hand; extends across the wrist</li> <li>· Also possible with silver in case of skin irritation</li> </ul>	<ul style="list-style-type: none"> <li>· Improved transfer of pressure to the back of the hand</li> <li>· Also possible with silver in case of skin irritation</li> </ul>
SoftCompress		Zipper	
			
<b>Hand pad</b>	<b>Zipper extension</b>	<b>Lymph lotion</b>	<b>Special detergent</b>
<ul style="list-style-type: none"> <li>· Improved transfer of pressure to the back of the hand</li> </ul>	<ul style="list-style-type: none"> <li>· Easier to open and close</li> </ul>	<ul style="list-style-type: none"> <li>· Moisturising lotion with 4 % urea for targeted skin care in cases of lymphoedema and lipoedema</li> </ul>	<ul style="list-style-type: none"> <li>· Particularly mild and economical concentrated detergent for gentle cleaning of compression garments</li> </ul>
Donning aids	Accessories		
			
<b>Arion Dactyna</b>	<b>Special gloves</b>		
<ul style="list-style-type: none"> <li>· For use with compression gloves with open fingers</li> </ul>	<ul style="list-style-type: none"> <li>· Anti-slip coating</li> <li>· Simplifies donning of compression garments</li> </ul>		







# Sleeves

- Available in
- Juzo Expert
  - Juzo Expert Silver
  - Juzo Expert Cotton
  - Juzo Expert Strong
  - Juzo Expert Strong Silver

CG



Model variants







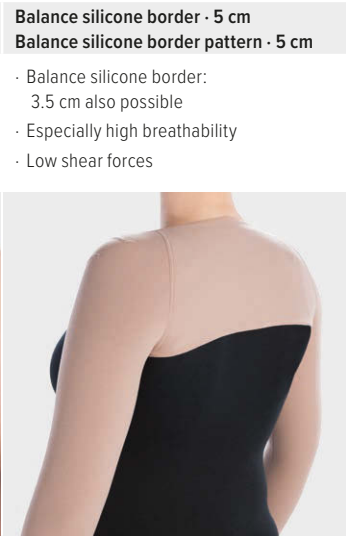
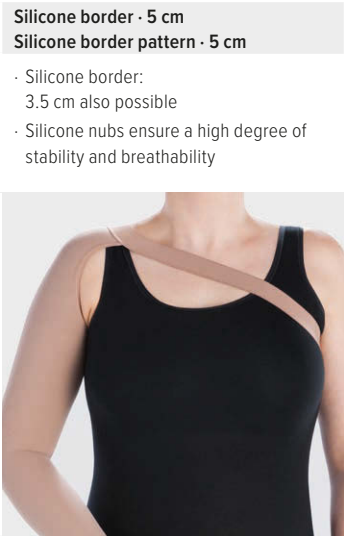




Sleeves (CG) with overheight

Shoulder and strap

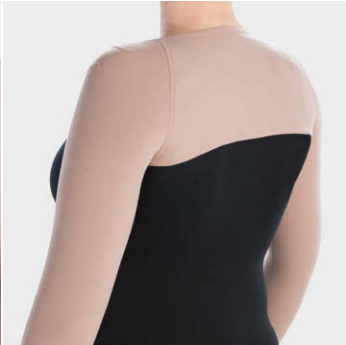One-piece sleeve and glove

Forearm cuff (CE)

Border · Fastening



Overheight / max. overheight at “cG”

Top border

Silicone border · 5 cm  
Silicone border pattern · 5 cm

Balance silicone border · 5 cm  
Balance silicone border pattern · 5 cm

Elastic border 3.5 cm

Bra fastening

Shoulder and strap

Bolero connection

- Lateral diagonal border
- Optimised pressure discharge to proximal

- Additional fastening necessary (e.g. adhesive lotion)

- Silicone border: 3.5 cm also possible
- Silicone nubs ensure a high degree of stability and breathability

- Balance silicone border: 3.5 cm also possible
- Especially high breathability
- Low shear forces

- Stable border
- Silicone-free


- Fastening option with Velcro fastener at the bra strap

- Alternative to silicone border

- Fastening option in the case of bilateral arm support
- With long and/or short sleeves
- Back portion without compression



## Extra options

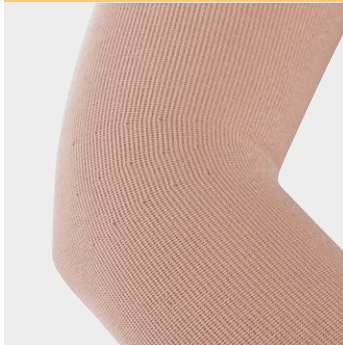**Anatomically flexed form at "cE"**

- 30° or 50° angulation possible
- In the case of a predominately angled elbow position

**Seam at the outside of the arm with CG**

- In the case of sensitivity in the crook of the elbow
- Only possible with anatomically angled shape 30°

**Adhesive borders**


- Additional adhesion on the upper arm
- Possible positioning: lateral at the outer side,, across

**Sewn-in silicone border on the inside**


- Only with overheight
- Increased stability and adhesion
- 3/4 sewn-in on the inside also possible
- Can be combined with silicone border at the top

**Bra fastening with silicone support**

- Additional hold support in the shoulder area

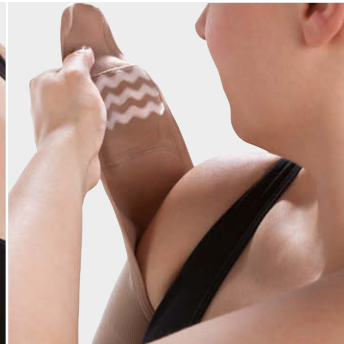**Bolero connection with silicone support**

- Additional hold support in the shoulder area

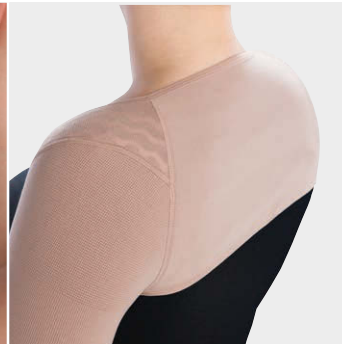**Tricot lining at "cE"**

- In the case of sensitive skin in the crook of the elbow
- Also possible with silver in case of skin irritation

**Incorporated pockets for pads**

- Customisation based on markings
- Also possible with silver in case of skin irritation





**Seams on the outside**

- Particularly flat seam touches the skin
- Suitable for sensitive skin

**Zipper**

- Individual positioning
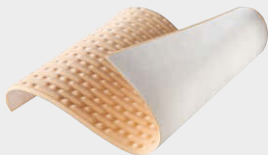
## Accessories · Sleeves

Lymphpads		SoftCompress	Fastening
			
<b>Lymphpad Line</b>	<b>Lymphpad Square</b>	<b>Elbow pad</b>	<b>Adhesive lotion</b>

- Stimulates micro circulation
- Synthetic latex, reverse side made of polyamide non-woven material
- Size 20 x 29.5 cm

- Stimulates micro circulation
- Synthetic latex, reverse side made of polyamide non-woven material
- Size 20 x 29.5 cm

- Optimization of the compression ratio in the crook of the elbow

- Special roll-on for securely holding stockings and sleeves

Care products		Donning aids	
			
<b>Lymph lotion</b>	<b>Special detergent</b>	<b>Slippie Arm</b>	<b>Arion EasySlide Arm</b>

- Moisturising lotion with 4% urea for targeted skincare in cases of lymphoedema and lipoedema

- Particularly mild and economical concentrated detergent for gentle cleaning of compression garments

- Very non-friction material

- Smooth, double-layered material

Accessories

<b>Special gloves</b>




- Anti-slip coating
- Simplifies donning of compression garments




# Thorax garments

Available in

- Juzo Expert
- Juzo Expert Silver

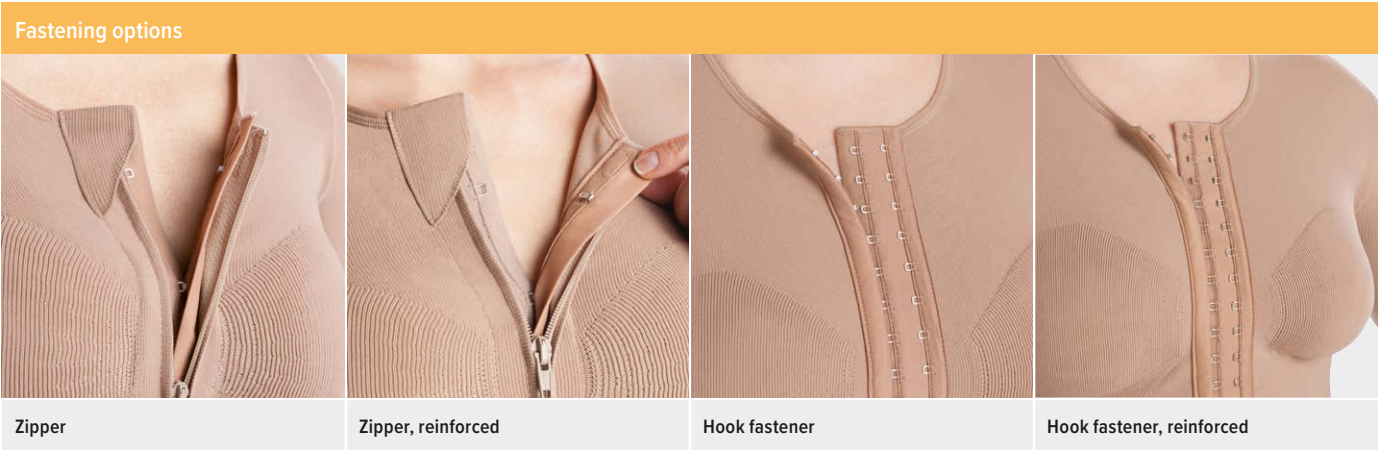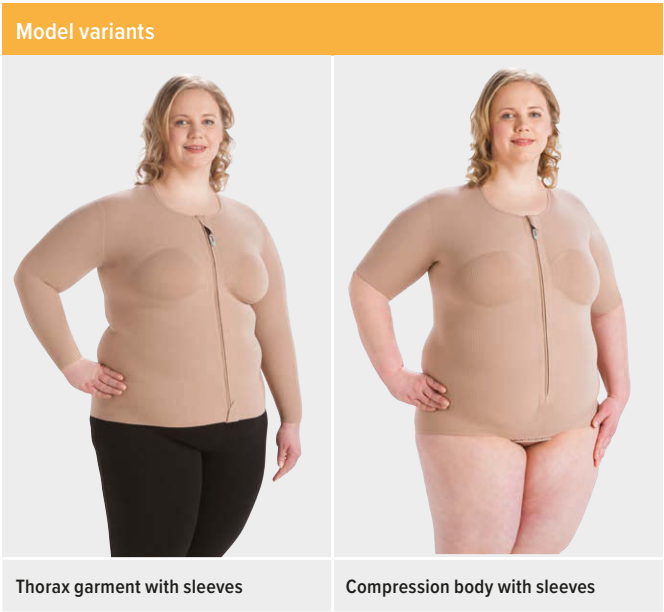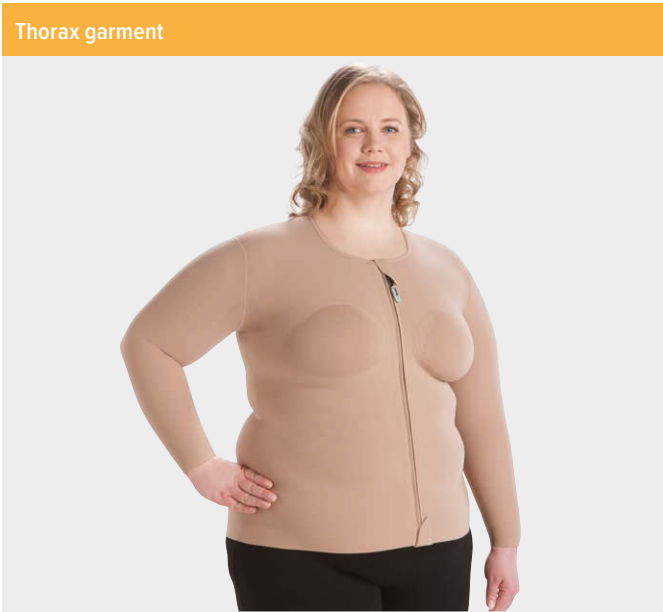


- |  |  |   |   |
|--|--|---|---|
| <b>Zipper</b>  | <b>Zipper, reinforced</b>  | <b>Hook fastener</b>  | <b>Hook fastener, reinforced</b>  |
| <ul style="list-style-type: none"><li>· With hook/eye underlap for easier closure</li><li>· Individual positioning is possible</li></ul> | <ul style="list-style-type: none"><li>· With reinforced hook/eye underlap</li><li>· Greater resilience for more extensive use</li><li>· Individual positioning is possible</li></ul> | <ul style="list-style-type: none"><li>· Second row of loops enables adjustment</li><li>· Individual positioning is possible</li></ul> | <ul style="list-style-type: none"><li>· Second row of loops enables adjustment</li><li>· Greater resilience for more extensive use</li><li>· Individual positioning is possible</li></ul> |

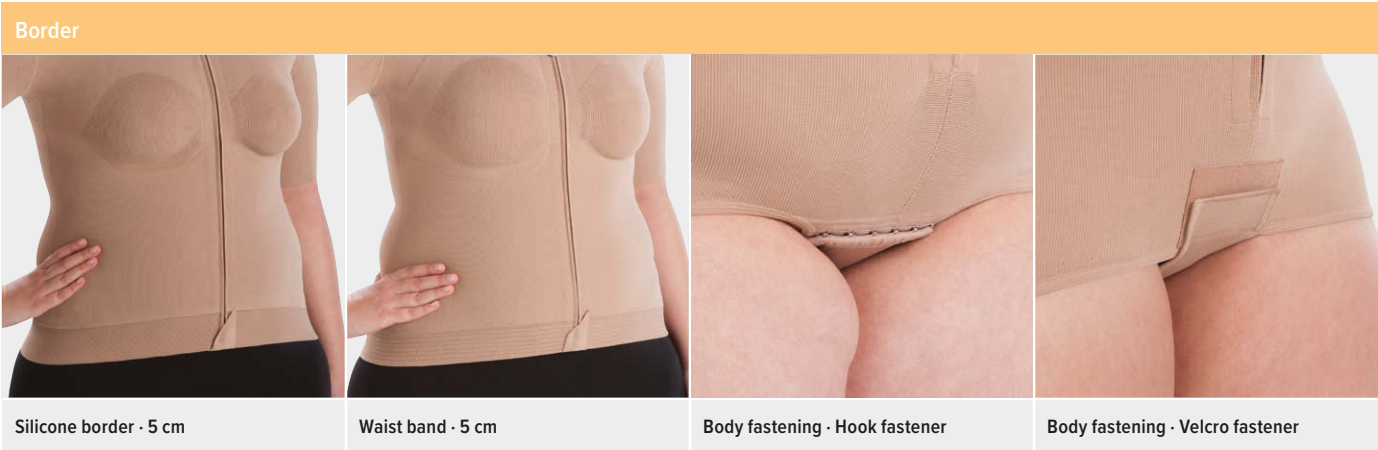

- |  |  |  |   |
|--|--|--|---|
| <b>Silicone border · 5 cm</b>  | <b>Waist band · 5 cm</b>   | <b>Body fastening · Hook fastener</b>                                | <b>Body fastening · Velcro fastener</b>   |
| <ul style="list-style-type: none"><li>· Flat, distal border</li><li>· With silicone nubs</li></ul> | <ul style="list-style-type: none"><li>· Flat, elastic, distal border</li></ul> | <ul style="list-style-type: none"><li>· Prevents riding up</li></ul> | <ul style="list-style-type: none"><li>· Prevents riding up</li><li>· Alternative to hook fastener</li></ul> |



## Extra options

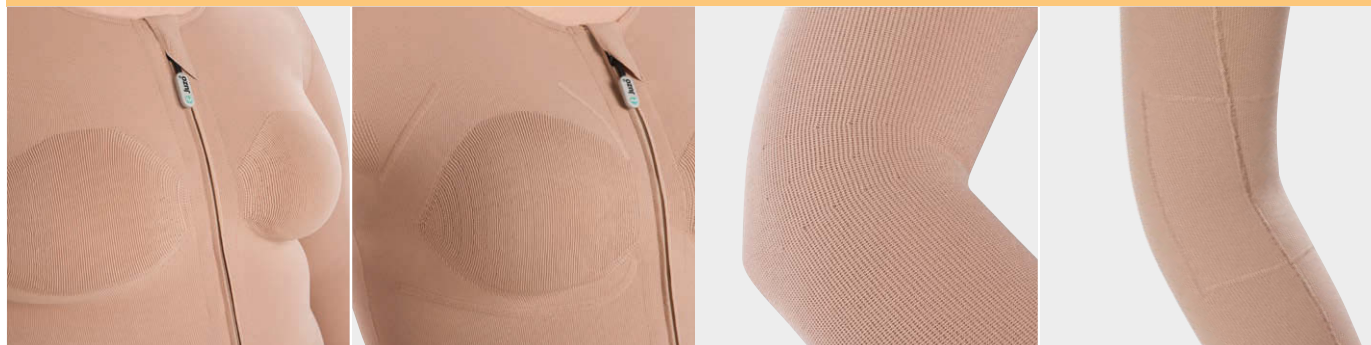**Knitted-in bra cups**

- Seamless bra cup
- Lightly compressive

**Prosthetics pocket**

- Sewn-in prosthetics pocket with three openings

**Anatomically flexed form at "cE"**

- 30° or 50° angulation possible
- In the case of a predominately angled elbow position

**Tricot lining at "cE"**

- In the case of sensitive skin in the crook of the elbow
- Also possible with silver in case of skin irritation

**Arm portion with integrated zipper**

- Individual positioning

**Incorporated pockets for pads**

- Customisation based on markings
- Also possible with silver in case of skin irritation

**Stand-up collar**

- In the case of oedema in the throat and neck regions or in the upper shoulder area (recommended in combination with a head bandage depending on the circumstances)





**Seams on the outside**

- Particularly flat seam touches the skin
- Suitable for sensitive skin



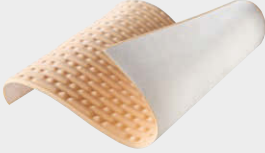

**Inelastic back zone**

- For adults
- Promotes upright body posture
- Prevents a relief posture from being adopted




## Accessories · Thorax garments

Lymphpads		ScarPad	
			
<b>Lymphpad Line</b>	<b>Lymphpad Square</b>	<b>ScarPad Extra</b>	<b>ScarPad Extra Strong</b>
<ul style="list-style-type: none"> <li>· Stimulates micro circulation</li> <li>· Synthetic latex, reverse side made of polyamide non-woven material</li> <li>· Size 20 x 29.5 cm</li> </ul>	<ul style="list-style-type: none"> <li>· Stimulates micro circulation</li> <li>· Synthetic latex, reverse side made of polyamide non-woven material</li> <li>· Size 20 x 29.5 cm</li> </ul>	<ul style="list-style-type: none"> <li>· Silicone patch for scars</li> </ul>	<ul style="list-style-type: none"> <li>· Silicone patch for scars</li> </ul>

SoftCompress		Zipper	Care products
			
<b>Breast pad</b>	<b>Elbow pad</b>	<b>Zipper extension</b>	<b>Special detergent</b>
<ul style="list-style-type: none"> <li>· For women</li> <li>· Targeted transfer of pressure in the breast and thorax area</li> </ul>	<ul style="list-style-type: none"> <li>· Optimisation of the compression ratio in the crook of the elbow</li> </ul>	<ul style="list-style-type: none"> <li>· Easier to open and close</li> </ul>	<ul style="list-style-type: none"> <li>· Particularly mild and economical concentrated detergent for gentle cleaning of compression garments</li> </ul>



Donning aids		Accessories
		
<b>Slippie Arm</b>	<b>Arion EasySlide Arm</b>	<b>Special gloves</b>
<ul style="list-style-type: none"> <li>· Very non-friction material</li> </ul>	<ul style="list-style-type: none"> <li>· Smooth, double-layered material</li> </ul>	<ul style="list-style-type: none"> <li>· Anti-slip coating</li> <li>· Simplifies donning of compression garments</li> </ul>




# Head garments

Available in


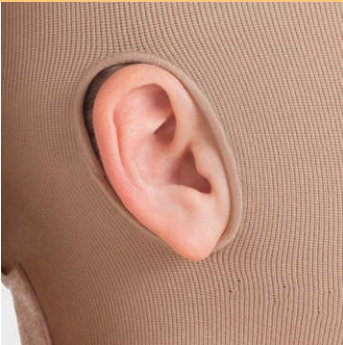

- Juzo Expert
- Juzo Expert Silver

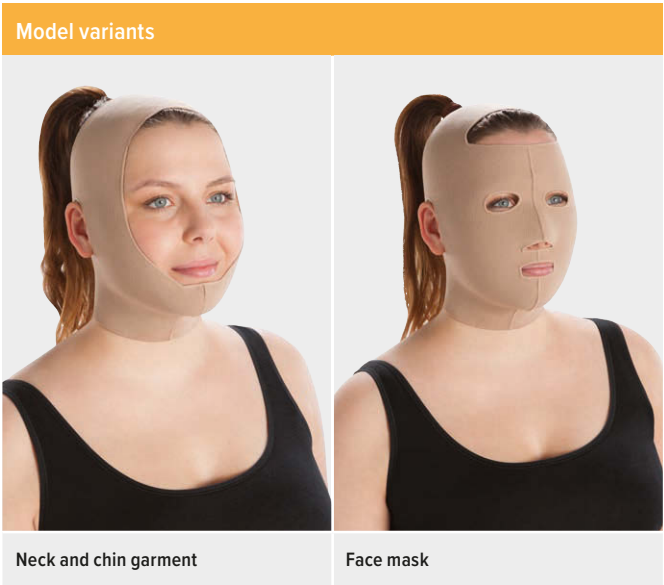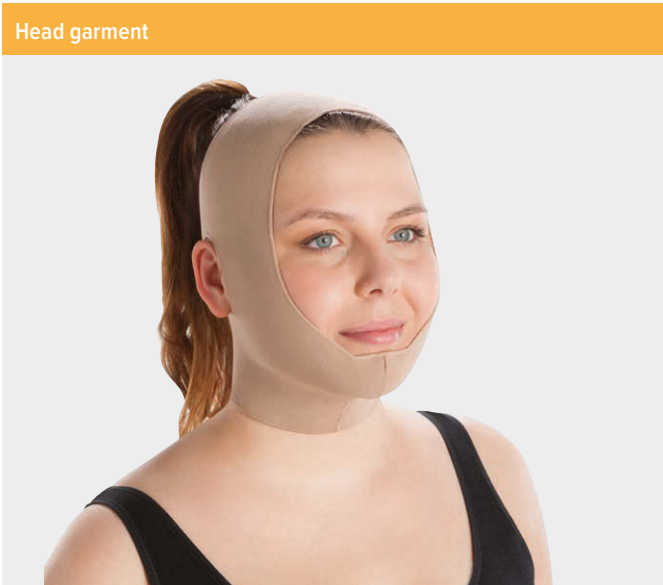

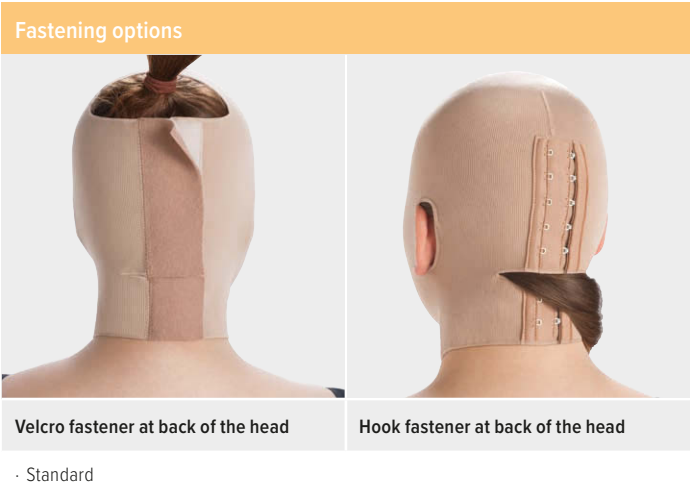


Head			
<b>Face mask</b> Closed forehead and back of the head	<b>Face mask</b> Open forehead and back of the head	<b>Face mask</b> Open forehead and back of the head	<b>Neck and chin garment</b> Open forehead and back of the head
<ul style="list-style-type: none"><li>· Cut-outs possible for eyes, nose, mouth, ears</li><li>· Knitted-in nose possible</li></ul>	<ul style="list-style-type: none"><li>· Cut-outs possible for eyes, nose, mouth, ears</li><li>· Knitted-in nose possible</li></ul>	<ul style="list-style-type: none"><li>· Free facial area</li><li>· Cut-outs possible for ears</li></ul>	<ul style="list-style-type: none"><li>· Cut-outs possible for ears</li></ul>





Extra options		
		
		
<b>Openings</b>		<b>Knitted-in nose</b>
<ul style="list-style-type: none"> <li>· Possible for eyes, mouth, ears and nose</li> </ul>		<ul style="list-style-type: none"> <li>· Compression in the nose area</li> </ul>

## Accessories · Head garments

Lymphpads		ScarPad
		
<b>Lymphpad Line</b>	<b>Lymphpad Square</b>	<b>ScarPad Light</b>
<ul style="list-style-type: none"> <li>· Stimulates micro circulation</li> <li>· Synthetic latex, reverse side made of polyamide non-woven material</li> <li>· Size 20 x 29.5 cm</li> </ul>	<ul style="list-style-type: none"> <li>· Stimulates micro circulation</li> <li>· Synthetic latex, reverse side made of polyamide non-woven material</li> <li>· Size 20 x 29.5 cm</li> </ul>	<ul style="list-style-type: none"> <li>· Silicone patch for scars</li> </ul>



Freedom in Motion





- Available in
- Juzo Expert
  - Juzo Expert Silver
  - Juzo Expert Cotton


## Stump garments (arm and leg)



### Model variants

- Below-knee stump
- Below-knee stump after Pirogoff amputation
- Above-knee stump
- Arm stump

### Border / Leg fastening

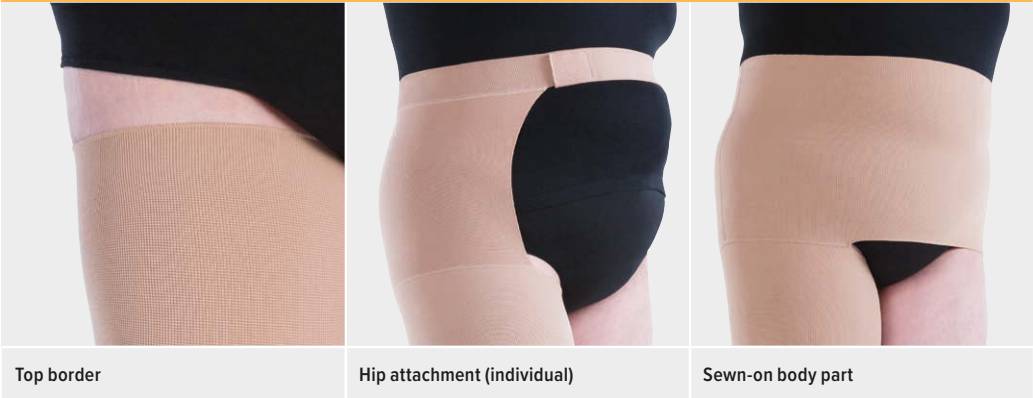

#### Top border

- Additional fastening necessary (e.g. Adhesive Lotion)

#### Hip attachment (individual)

- Additional silicone border for better fixation is possible
- Worn on the left or right

#### Sewn-on body part

- Alternative fastening option with compression in the stomach area

### Border / Arm fastening



#### Top border

- Additional fastening necessary (e.g. Adhesive Lotion)

#### Bra fastening

- Fastening option with Velcro fastener at the bra strap



#### Shoulder and strap

- Alternative to silicone border








#### Opening for prosthetic

- 3-cm opening to the end of the stump

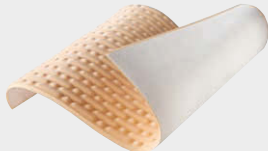







Silicone borders	
	
<b>Silicone border - 3.5 cm / 5 cm</b>	<b>Balance silicone border - 3.5 cm / 5 cm</b>
<ul style="list-style-type: none"> <li>· 3.5 cm can be used with below-knee and arm stump shrinkers</li> <li>· 5 cm can be used with below-knee and above-knee stump shrinkers</li> <li>· Silicone nubs ensure a high degree of stability and breathability</li> </ul>	<ul style="list-style-type: none"> <li>· 3.5 cm can be used with below-knee and arm stump shrinkers</li> <li>· 5 cm can be used with below-knee and above-knee stump shrinkers</li> <li>· Especially high breathability</li> <li>· Low shear forces</li> </ul>

## Accessories · Stump garments (arm and leg)







Lymphpads		ScarPad	
			
<b>Lymphpad Line</b>	<b>Lymphpad Square</b>	<b>ScarPad Extra</b>	<b>ScarPad Extra Strong</b>
<ul style="list-style-type: none"> <li>· Stimulates micro circulation</li> <li>· Synthetic latex, reverse side made of polyamide non-woven material</li> <li>· Size 20 x 29.5 cm</li> </ul>	<ul style="list-style-type: none"> <li>· Stimulates micro circulation</li> <li>· Synthetic latex, reverse side made of polyamide non-woven material</li> <li>· Size 20 x 29.5 cm</li> </ul>	<ul style="list-style-type: none"> <li>· Silicone patch for scars</li> </ul>	<ul style="list-style-type: none"> <li>· Silicone patch for scars</li> </ul>
Care products	Donning aids	Accessories	
			
<b>Special Detergent</b>	<b>Arion Proth-Pro</b>	<b>Special gloves</b>	
<ul style="list-style-type: none"> <li>· Particularly mild and economical concentrated detergent for gentle cleaning of compression garments</li> </ul>	<ul style="list-style-type: none"> <li>· Donning aid for thigh prostheses with suction socket</li> </ul>	<ul style="list-style-type: none"> <li>· Anti-slip coating</li> <li>· Simplifies donning of compression garments</li> </ul>	

# Additional products

## The perfect supplements

SoftCompress					
					
<b>Breast pad</b>	<b>Genital pad for women</b>	<b>Pad for the crook of the elbow</b>	<b>Pad for the back of the knee</b>	<b>Pad for the ankle</b>	<b>Hand pad</b>
<ul style="list-style-type: none"> <li>· For women</li> <li>· Targeted transfer of pressure in the breast and thorax area</li> </ul>	<ul style="list-style-type: none"> <li>· For insertion into a compression pantyhose</li> <li>· Targeted pressure optimisation in the case of genital oedema; ideal pressure distribution</li> </ul>	<ul style="list-style-type: none"> <li>· Optimisation of the compression ratio in the crook of the elbow</li> </ul>	<ul style="list-style-type: none"> <li>· Optimisation of the compression ratio at the back of the knee</li> </ul>	<ul style="list-style-type: none"> <li>· For improving pressure in the ankle area</li> <li>· With a skin-friendly cotton cover</li> </ul>	<ul style="list-style-type: none"> <li>· Improved transfer of pressure to the back of the hand</li> </ul>

Further information is available in the brochure Juzo special products for decongestion.

Lymphpads		Pressure pads			Liner stocking
					
<b>Lymphpad Line</b>	<b>Lymphpad Square</b>	<b>Pressure pad (Dr. Rotter)</b>	<b>Malleoli pads</b>	<b>Pressure pad, long</b>	<b>Silver liner stocking</b>
<ul style="list-style-type: none"> <li>· Stimulates micro circulation</li> <li>· Synthetic latex, reverse side made of polyamide non-woven material</li> <li>· Size 20 x 29.5 cm</li> </ul>	<ul style="list-style-type: none"> <li>· Stimulates micro circulation</li> <li>· Synthetic latex, reverse side made of polyamide non-woven material</li> <li>· Size 20 x 29.5 cm</li> </ul>	<ul style="list-style-type: none"> <li>· For improving pressure in the ankle area</li> <li>· Made of carbosan moulded foam</li> <li>· With textile elastic cover or cover containing silver</li> </ul>	<ul style="list-style-type: none"> <li>· For improving pressure in the ankle area</li> <li>· Made of silicone; soft and flat</li> <li>· Uncovered or with textile elastic cover or cover containing silver</li> </ul>	<ul style="list-style-type: none"> <li>· Improved transfer of pressure to the back of the hand; extends across the wrist</li> <li>· Also possible with silver in case of skin irritation</li> </ul>	<ul style="list-style-type: none"> <li>· With antibacterial silver thread</li> <li>· Fixation of wound dressings and pads</li> </ul>

# Practical tips for caregivers

## Check the fit regularly

Compression garments must fit perfectly. However, the way a garment fits can change over time if the patient gains or loses weight, or if circumferences change (e.g. oedema reduction).

Caregivers should therefore regularly check to make sure garments still fit properly and then have alterations made or recommend that a new garment be produced if necessary.

## Explain how to take care of the product

Compression garments should be washed every day at 40 °C in order to allow the material to regenerate. It is recommended that garments be washed with Juzo Special Detergent and that a fabric softener not be used.

## Get patients to take care of their skin daily

Dry skin can be kept supple by massaging it and applying a cream every day. Skin should always be kept clean and elastic in order to prevent inflammation.

## Explain the wearing period

Compression garments should be worn as prescribed by a physician, as this is the only way to guarantee an optimum treatment outcome. Garments must be worn consistently every day or for the periods prescribed by a physician.

In order to ensure proper hygiene and material regeneration, caregivers should instruct patients to change their compression garments every day. Ideally, patients should therefore be issued more than one compression garment.

## Instruct patients to avoid excessive exposure to sunlight

Exposure to direct sunlight should be avoided in order to prevent skin changes and pigment disorders.

# Case study

## Compression therapy with Juzo Expert and Juzo Expert Strong

Ursula K. (born in 1954) was diagnosed with cervical cancer at the age of 39. Following a Wertheim-Meigs operation and the removal of more than 30 lymph nodes and a lymph cyst in her lower abdomen, Ursula K. quickly developed lymphoedema in both legs. She has been receiving treatment for this in the form of MLD and compression therapy since 1993. She was initially given a circular knit, but the results were less than optimal. Ursula K. came across Juzo Expert by coincidence after about three years of treatment. Since that time, she has been wearing Juzo Expert and Juzo Expert Strong.



Secondary lymphoedema in both legs following several surgeries to address cervical cancer



Treatment example: Pantyhose (AT), Juzo Expert



Treatment example – multiple products: Stockings (AG), left leg in Juzo Expert Strong (CCL 3), right leg in Juzo Expert (CCL 2), with Juzo Expert leggings (CCL 2) on top