



Special-purpose garments

Customised compression products for lymphology – for maximum therapeutic success with more comfort




Customer Service department

Juzo – Always there for you

Engaging, friendly and competent – that is how our customers describe us. We look after our customers with a smile and genuine interest in their wishes. Our goal is to answer their needs as best as possible, which is why we are always on the lookout for new, optimised solutions. In these efforts, we rely on our experience and our unique expertise, which we continue to develop further every day. We are always there for you with help and advice – not just to fulfil our own claims – but to assist you in supporting the therapy success of your patients with the right products. We are looking forward to your call and finding a solution that's right for you – because “Can't be done” is never an option for us!

Our Customer Service department offers:

- + general information
- + individual product advice
- + order acceptance
- + feedback on the status of your order
- + acceptance of alteration or repair orders
- + advice on questions about compression therapy

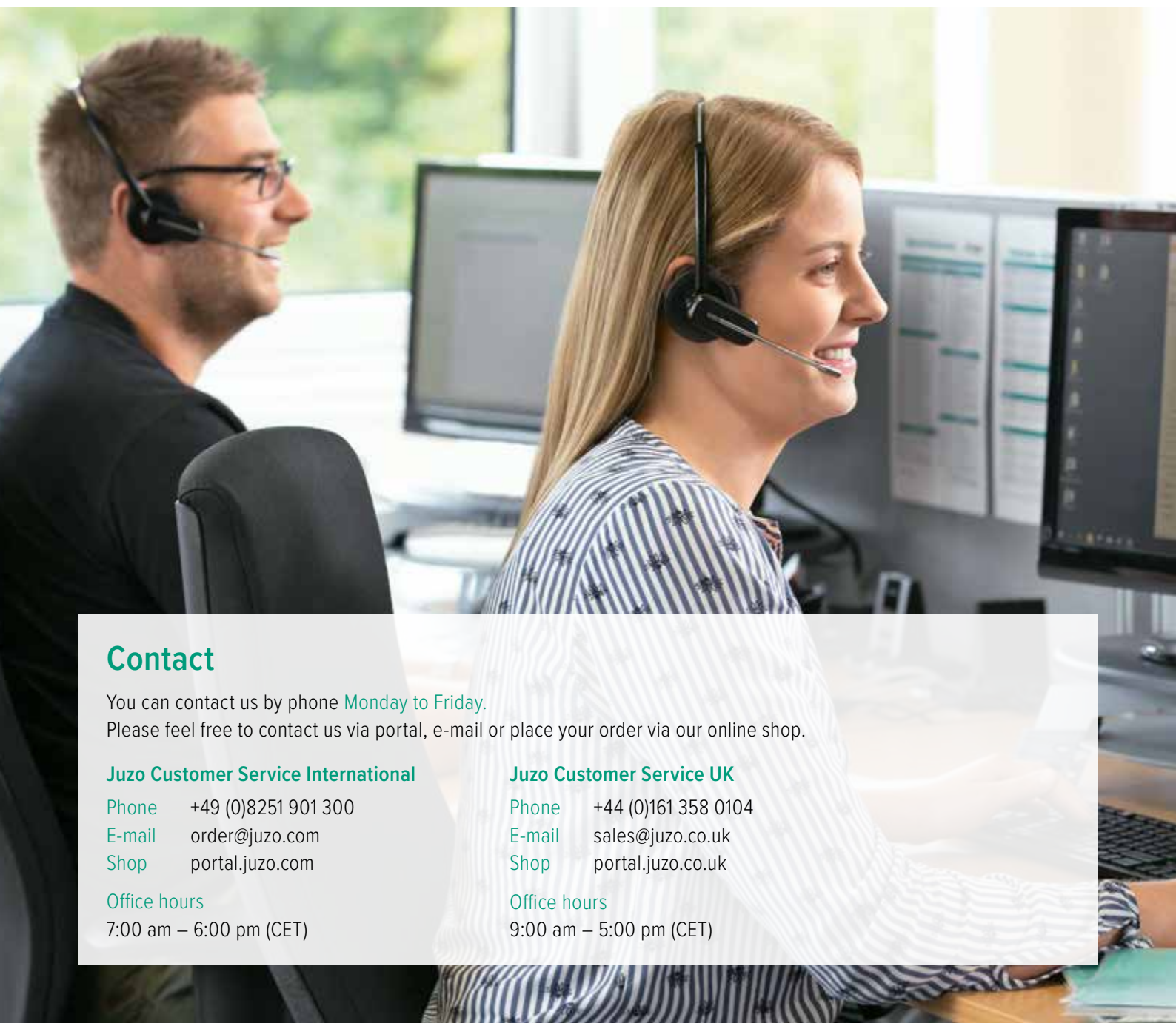

Contact

You can contact us by phone [Monday to Friday](#).
Please feel free to contact us via portal, e-mail or place your order via our online shop.

Juzo Customer Service International

Phone +49 (0)8251 901 300
E-mail order@juzo.com
Shop portal.juzo.com

Office hours

7:00 am – 6:00 pm (CET)

Juzo Customer Service UK

Phone +44 (0)161 358 0104
E-mail sales@juzo.co.uk
Shop portal.juzo.co.uk

Office hours

9:00 am – 5:00 pm (CET)

Here's how to find the right product quickly

General information

Effective oedema therapy	04
Compression systems for decongestion	05
Lymphology expert	06
Practical tips for caregivers	47
Case study	47

Product overview

Toe-portions	09
Below-knee stockings · AD	11
Thigh-high stockings · AG	15
Pantyhose · AT	19
Gloves	27
Sleeves	31
Thorax garments	35
Head garments	39
Stump garments (arm and leg)	43
Diverse combination options	46

Effective oedema therapy


The components

Components of oedema therapy

- + Manual lymphatic drainage (MLD)
- + Compression therapy
- + Decongesting movement and breath therapy
- + Skin care and skin rehabilitation
- + Education and training for individual self-therapy

Goals of oedema therapy

- + Improvement of lymph flow
- + Loosening of hardened, fibrotic tissue
- + Reduction of connective tissue increase
- + Improvement of mobility in the affected limbs in order to increase effectiveness of the muscle and joint pump
- + For lipoedema: pain alleviation, reduction of haematoma and oedema

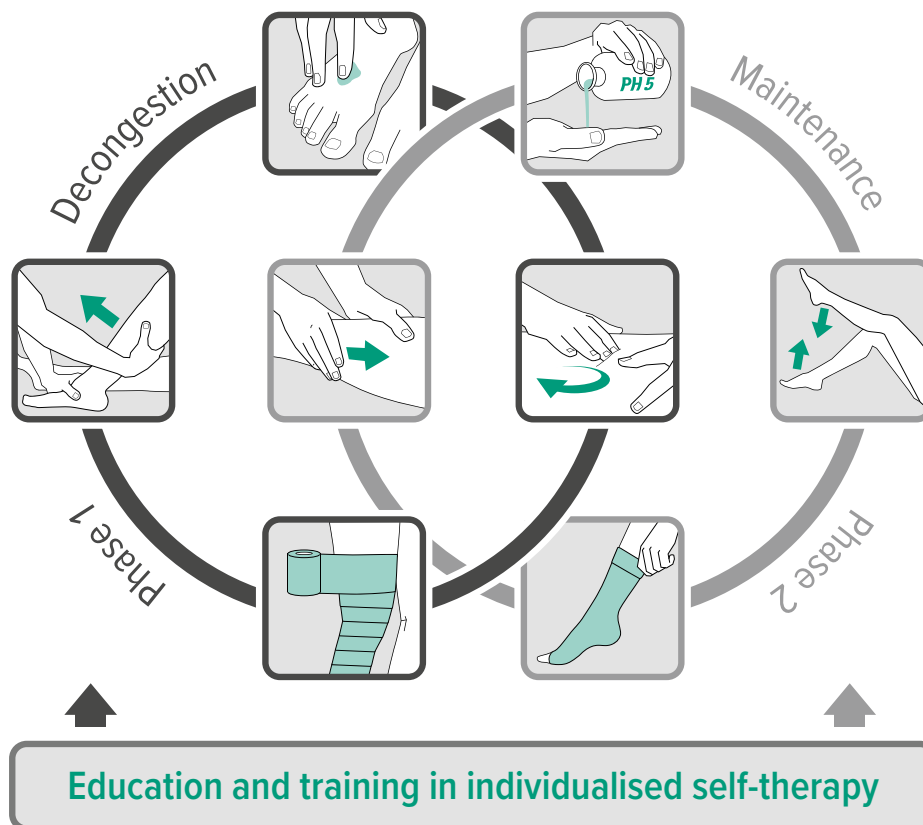

Phase 1: Decongestion phase

Using manual lymphatic drainage (MLD), the transport capacity of the lymphatic system is increased in order to enable mobilisation and removal of protein-rich oedema fluid (lymph). The decongested limb is then compressed with compression bandages or supports (e.g. Juzo SoftCompress). This prevents the return flow of lymphatic fluid and reduces the increase in connective tissue.

Phase 2: Maintenance phase

In the second phase, the applications of MLD are reduced and compression bandages or supports are replaced by flat-knit compression garments (e.g. Juzo Expert/ Juzo Expert Strong). This optimises and maintains long-term treatment success.

In both phases, consistent care protects the skin from drying out and minimises the risk of infection. Physical activities and decongestive exercises also help to stimulate lymphangiomotoricity. The active engagement of the patient is important for the long-term success of the treatment.

Compression systems for decongestion

Juzo SoftCompress and adjustable compression systems

Compression bandages

Juzo SoftCompress can significantly increase the success of decongestive therapy. Bandaging aids and additional products feature a particularly skin-friendly cotton surface, as opposed to frequently used synthetic materials. This provides the patient with pleasant wearing comfort. Vertical foam channels form a unique surface structure that supports lymphatic drainage. In addition, Juzo SoftCompress bandaging aids make it easy for patients to use the bandages, therefore ensuring the long-term success of the treatment.



Adjustable compression systems

Adjustable compression systems can be used in decongestive therapy as an efficient alternative to conventional compression rolls. These innovative products enable health care professionals to promote treatment success whilst saving time and money. They can be adapted to the individual patient – even if circumferences change during treatment. Easy handling of the hook and loop fasteners means that patients can take care of the adjustment themselves to ensure the compressive effect is perfectly maintained, even with increased decongestion. If there are open wounds in the area to be treated, the affected area must be covered with appropriate wound dressing before the compression system can be applied. To protect and secure wound dressings, we recommend wearing a liner stocking (e.g. Juzo Liner).

Juzo ACS Light



Juzo Compression Wrap



Juzo SoftCompress and adjustable compression systems can also be used together. Further information is available in the brochure Juzo “Compression Systems for decongestion”.



Lymphology expert

Versatile compression products from head to toe

Lymphology applications

Lymphology treatment approaches mainly utilise flat-knitted products with seams, which, along with lymphatic drainage, form the basis for all types of oedema therapies. Compression garments ensure the long-term success of decongestive therapies and are generally indispensable in lymphology. From the foot-toe-portion to stockings, pantyhose and face masks – treatment covers all areas of the body. The flat-knitted products are precisely tailored to the patient's body measurements and, depending on the area of application, are also available in different versions. The combination of various materials here – e.g. with cotton or silver – makes it possible for Juzo to offer products for diverse patient requirements.

How they work

The strong knitted fabrics stand through special features that include a massage effect that promotes lymph drainage. The fabrics also do not yield to the oedema. During movement, a high working pressure is generated that enables optimal compression of the tissue affected by the oedema, which is ideal for a lymphology treatment approach.

Convincing product benefits

- + Optimal wearing comfort for patients due to the particularly flat and elastic seams
- + The homogeneous, soft and smooth material ensures patient mobility is maintained
- + Exceptionally skin-friendly due to the double-layered textile-yarn winding in the compression threads
- + Compression garments can be worn normally every day, thanks to the use of latex-free materials that are more resistant to sunlight and ointments and are more resilient, yet more breathable as well
- + Easy to put on and take off, thanks to the knitted fabric's fine mesh surface structure and optional integration of additional fasteners (hook fastener, zipper, touch fastener)

Juzo Expert

For individual oedema treatment

Juzo Expert offers maximum freedom of movement and is particularly pleasant to wear. The knitted fabric is perfect for flat knit beginners and whole body treatments.



Juzo Expert

Juzo Expert Strong

Strong for pronounced oedema

Juzo Expert Strong is characterised by its high working pressure and activating massage effect that stimulates lymphatic drainage. The firm and strong fabric provides effective support in the treatment of pronounced lymphoedema and lipoedema.



Juzo Expert Strong

Compression garments with silver thread

The silver content, which is attuned to the skin flora, restores the balance of inflamed or irritated skin and protects against further infections. The antibacterial effect is caused by released silver ions that are effective in combating a broad range of bacteria such as *Staphylococcus aureus*. Wearing the compression garments containing silver reduces the bacteria on the skin and itching – preventing the onset of skin infections. The bactericidal effect also reduces the formation of substances that produce body odour. The silver knitted fabric also helps to provide exceptional wearing comfort through the cooling or warming properties of the material.

Please visit juzo.com/silver for more information on the Juzo compression garments with silver thread.

Compression garments with cotton

Juzo Expert Cotton and Juzo Expert Strong Cotton are particularly soft and smooth thanks to their rich cotton content. The garments are particularly suited to sensitive skin as the skin only comes into contact with the pure natural fibre. The knitted fabrics are effective thanks to their high wearing comfort and pleasant skin climate. The soft material also makes it easy to put on and take off the garment. The high-quality cotton fibres lend the garments a particularly glossy look, which really appeals to fashion-conscious wearers.

Please visit juzo.com/cotton for more information.

Colours

Available for Juzo Expert and Juzo Expert Strong (CCL 1–3)*



Sugar



Sesame



Almond



Cinnamon



Cacao



Poppy seed



Blueberry



Black pepper



Trend Colours

* Silver, Cotton and CCL 4 available in colour Almond.

Available for Juzo Expert



Batik Collection

All standard colours (except Sugar and Almond) and Trend Colours can be batiked in black or white.



Dip Dye Collection

Available in colour Poppy seed and Blueberry in CCL 1–2 as AD, AF, AG and sleeves in custom-made.

For current Trend Colours, Batik Collection and Dip Dye Collection see Juzo "Productoverview Lymphology" or juzo.com





Toe-portions

Toe-portions

- Available in
- Juzo Expert
 - Juzo Expert Silver
 - Juzo Expert Cotton
 - Juzo Expert Strong
 - Juzo Expert Strong Silver



- Seams on the outside
- Particularly flat seam touches the skin
 - Suitable for sensitive skin
 - Also available with open toes



Below-knee stockings · AD

- Available in
- Juzo Expert
 - Juzo Expert Silver
 - Juzo Expert Cotton
 - Juzo Expert Strong
 - Juzo Expert Strong Silver
 - Juzo Expert Strong Cotton

AD




Model variants


AD



AB/AB'


BD


Toes



Ball stub/diagonal foot with open toes

- For oedema on the top of the foot


Ball stub/diagonal foot with closed toes

- For oedema on the top of the foot


Open toes








Closed toes

- Very elastic
- High degree of wearing comfort


Sewn-on foot-toe-portion

- No overlapping of stocking and foot-toe-portion
- Little toe (5) can be open

Border / Fastening

			
Overheight/max. overheight	Top border	Silicone border · 3.5 cm Silicone border pattern · 3.5 cm	Balance silicone border · 3.5 cm Balance silicone border pattern · 3.5 cm
<ul style="list-style-type: none"> · Lateral diagonal border · Optimised pressure discharge to proximal 	<ul style="list-style-type: none"> · Additional fastening option available if needed (e.g. adhesive lotion) 	<ul style="list-style-type: none"> · Silicone border: 5 cm also possible · Silicone nubs ensure a high degree of stability and breathability 	<ul style="list-style-type: none"> · Balance silicone border: 5 cm also possible · Especially high breathability · Low shear forces
			
Lace silicone border · 5 cm	Elastic border · 3.5 cm		
<ul style="list-style-type: none"> · Circumferential silicone strips ensure particularly good adhesion 	<ul style="list-style-type: none"> · Stable border · Silicone-free 		

Extra options			
			
Additional instep stitching	Silicone stopper	Tricot lining at "cY"	Integrated pressure pad (Dr. Rotter)
<ul style="list-style-type: none"> · Y-stitching /90° foot · Less creasing in the stretched/instep area, thanks to the anatomically shaped heel 	<ul style="list-style-type: none"> · Bi-elastic material · Possible positioning: lateral at the outer side, across 	<ul style="list-style-type: none"> · For sensitive skin in the stretched/instep area · Also possible with silver in case of skin irritation 	<ul style="list-style-type: none"> · Improvement of compression in the retromalleolar area · Also possible with silver in case of skin irritation · Malleoli pads made of silicone also available as an alternative
			
Sewn-in silicone border on the inside	Incorporated pockets for pads	Seams on the outside	Zipper
<ul style="list-style-type: none"> · Only possible with overheight · 3/4 sewn-in on the inside also possible · Can be combined with silicone border at the top 	<ul style="list-style-type: none"> · Customisation based on markings · Also possible with silver in case of skin irritation 	<ul style="list-style-type: none"> · Particularly flat seam touches the skin · Suitable for sensitive skin 	<ul style="list-style-type: none"> · Standard in the seam · Individual positioning is possible
			
Silicone coating on the sole of the foot	Place lace seams outwards		
<ul style="list-style-type: none"> · Anti-slip coating 	<ul style="list-style-type: none"> · Suitable for sensitive toes · Possible with closed toes 		



Thigh-high stockings · AG

- Available in
- Juzo Expert
 - Juzo Expert Silver
 - Juzo Expert Cotton
 - Juzo Expert Strong
 - Juzo Expert Strong Silver
 - Juzo Expert Strong Cotton



Toes

Ball stub/diagonal foot with open toes

· For oedema on the top of the foot

Ball stub/diagonal foot with closed toes

· For oedema on the top of the foot

Open toes

· Very elastic
· High degree of wearing comfort




Closed toes



Border / Fastening

			
Side overheight/max. overheight <ul style="list-style-type: none"> · Lateral diagonal border · Optimised pressure discharge to proximal 	Overheight at the front of the leg <ul style="list-style-type: none"> · Raised for a better fit for patients with ventrolateral thigh oedema 	Top border <ul style="list-style-type: none"> · Additional fastening options available if needed (e.g. adhesive lotion) 	Silicone border · 5 cm Silicone border pattern · 5 cm <ul style="list-style-type: none"> · Silicone nubs ensure a high degree of stability and breathability
			
Balance silicone border · 5 cm Balance silicone border pattern · 5 cm <ul style="list-style-type: none"> · Especially high breathability · Low shear forces 	Lace silicone border · 5 cm <ul style="list-style-type: none"> · Circumferential silicone strips ensure particularly good adhesion 	Hip attachment · individual <ul style="list-style-type: none"> · Additional silicone border for better fixation is possible · Worn on the left or right 	Hip attachment · pair <ul style="list-style-type: none"> · Opened at the front and back · Available with a strap on request (opened at the front) · Additional silicone border is possible
			
Sewn-on body part <ul style="list-style-type: none"> · Alternative fastening option with compression in the stomach area 			


Extra options

			
Additional instep stitching <ul style="list-style-type: none"> · Y-stitching / 90° foot · Less creasing in the stretched/instep area, thanks to the anatomically shaped heel 	Anatomically flexed form at "cE" <ul style="list-style-type: none"> · 30° or 50° angulation possible · In the case of a predominately angled knee position 	Silicone stopper <ul style="list-style-type: none"> · Bi-elastic material · Possible positioning on the thigh: front on the side, lengthwise; across on the outside 	
			
Sewn-in silicone border on the inside <ul style="list-style-type: none"> · Only possible with overheight · 3/4 sewn-in on the inside also possible · Can be combined with silicone border at the top 	Tricot lining at "cY" <ul style="list-style-type: none"> · For sensitive skin in the stretched/instep area · Also possible with silver in case of skin irritation 	Tricot lining at "cE" <ul style="list-style-type: none"> · For sensitive skin at the back of the knee · Also possible with silver in case of skin irritation 	Integrated pressure pad (Dr. Rotter) <ul style="list-style-type: none"> · Improvement of compression in the retromalleolar area · Also possible with silver in case of skin irritation · Malleoli pads made of silicone available
			
Incorporated pockets for pads <ul style="list-style-type: none"> · Customisation based on markings · Also possible with silver in case of skin irritation 	Seams on the outside <ul style="list-style-type: none"> · Particularly flat seam touches the skin · Suitable for sensitive skin 	Porosa starting at "cD" <ul style="list-style-type: none"> · Compression reduction starting at "cD" 	Zipper <ul style="list-style-type: none"> · Standard in the seam · Individual positioning is possible
			
Bilateral dual spiral stays and/or patella ring <ul style="list-style-type: none"> · Integrated knee support 	Silicone coating on the sole of the foot <ul style="list-style-type: none"> · Anti-slip coating 	Place lace seams outwards <ul style="list-style-type: none"> · Suitable for sensitive toes · Possible with closed toes 	



Pantyhose · AT


- Available in
- Juzo Expert
 - Juzo Expert Silver
 - Juzo Expert Cotton
 - Juzo Expert Strong
 - Juzo Expert Strong Silver
 - Juzo Expert Strong Cotton

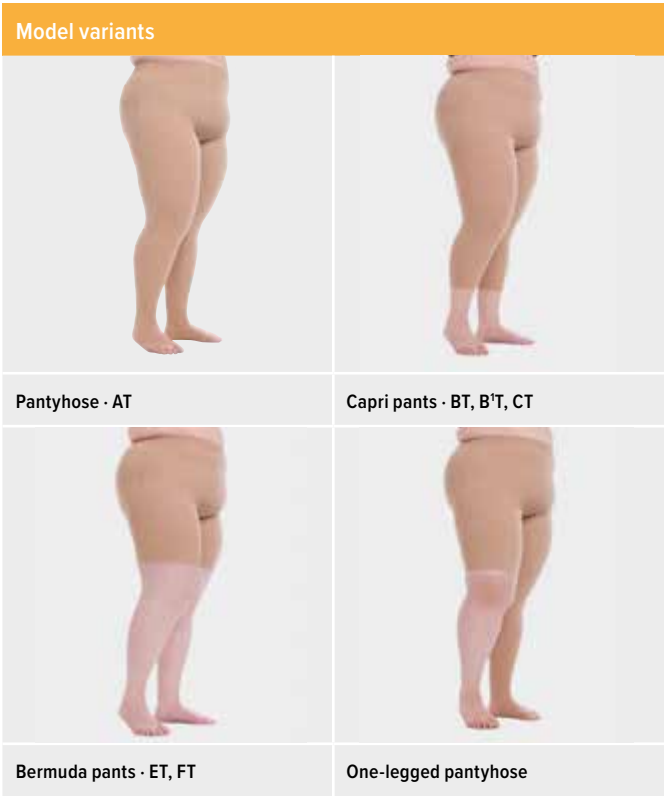

Border / Fastening

**Oblique version**

- Standard
- In case of frontal oedema development
- In case of a flat inguinal orientation
- Continuous seam gradient

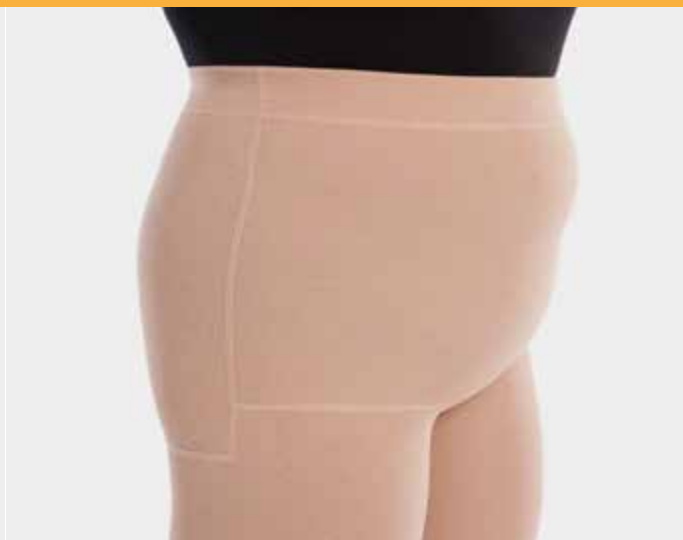**Box type**

- For large circumference dimensions
- In case of lateral oedema development
- In case of pronounced hip and buttock shapes
- Particular wearing comfort when sitting

**Slip form**

- In case of less pronounced stages of oedema and body shapes
- In case of a steep inguinal orientation
- In case of a flat abdominal area
- Preferred for lipoedema stadium 1 and 2

**Mat fit (pregnancy fastening)**

- Body part in the back area CCL 1
- Body part without compression (grows with the body)
- Touch fastener can be on the right or left
- Seam gradient as oblique version
- Minimum body part length: 34 cm

Standard combinations (all other combinations are possible)**AT body parts in general**

AT	CCL			
	1	2	3	4
Leg parts	1	2	3	4
Body part	1	2	2	2

Border / Fastening

**Top border**

- Soft border

Adjustable waistband

- Retractable border enables individual adjustment options

Waist band

- Flat and elastic border
- The border of the knitted fabric will not fold over

Waist band with touch fastener

- Flat and elastic border enables adjustment

**Silicone border · 5 cm**

- Flat border
- Additional fastening with silicone nubs

Body part with zipper

- In the middle as standard
- Individual positioning is possible

Body part with hook fastener

- In the middle as standard
- Individual positioning is possible

Body part with touch fastener

- In the middle as standard
- Individual positioning is possible

Extra options

**Gusset**

- Knitted
- Compressive gusset also possible

Small gusset

- Beneficial in the case of large thigh circumferences
- Compressive gusset also possible

Mesh gusset

- Better air circulation

Small mesh gusset

- Beneficial in the case of large thigh circumferences
- Better air circulation

**Fly**

- For men
- Horizontal design

Scrotum

- Optimum fit in the genital area for men
- Also available in combination with a fly
- Mesh material also possible


Stoma opening

- Customisation based on markings

Open crotch

- Without gusset
- Not recommended in the case of adipose tissue

Extra options

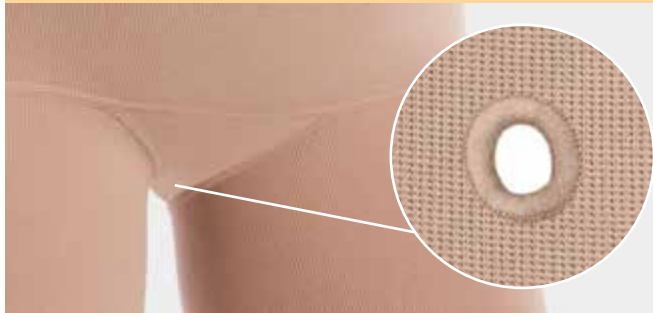**NEW Gusset with catheter opening****NEW Gusset with slot**


· For incontinence aids

**Additional instep stitching**

- Y-stitching/90° foot
- Less creasing in the stretched/instep area, thanks to the anatomically shaped heel

**Anatomically flexed form at "cE"**

- 30° or 50° angulation possible
- In the case of a predominately angled knee position

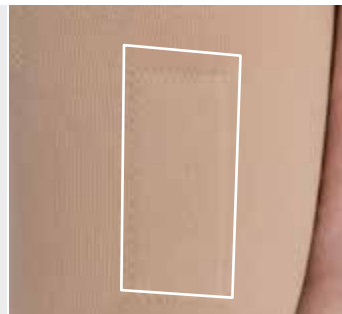**Adhesive borders**

- Bi-elastic material
- Placement options on thigh: lateral at the outer side, lengthwise at the front
- Placement options in body part: front on the side, lengthwise; across on the outside

**Sewn-in 3/4 silicone border on the inside**

- On the thigh
- Increased stability and adhesion
- Only possible for body part in the oblique version

**Tricot lining at "cY"**

- For sensitive skin in the stretched/instep area
- Also possible with silver in case of skin irritation

**Tricot lining at "cE"**

- For sensitive skin at the back of the knee
- Also possible with silver in case of skin irritation

**Integrated pressure pad (Dr. Rotter)**

- Improvement of compression level in the retromalleolar area
- Also possible with silver in case of skin irritation
- Malleoli pads made of silicone also available as an alternative

**Incorporated pockets for pads**

- Customisation based on markings
- Also possible with silver in case of skin irritation

**Seams on the outside**

- Particularly flat seam touches the skin
- Suitable for sensitive skin

**Zipper**





- Standard in the seam
- Individual positioning is possible

**Bilateral dual spiral stays and/or patella ring**





- Integrated knee support

Extra options	
	
Silicone coating on the sole of the foot <ul style="list-style-type: none"> · Anti-slip coating 	Place lace seams outwards <ul style="list-style-type: none"> · Suitable for sensitive toes · Possible with closed toes

Additional products · AD, AG and AT

Pads		Lymphpads	
			
Pressure pad (Dr. Rotter) · Art. 9400/9401 (antibacterial, containing silver) <ul style="list-style-type: none"> · For improving pressure in the ankle area · Made of carbosan moulded foam 	Malleoli pads · Art. 9408 Without cover <ul style="list-style-type: none"> · Art. 9409 Covered with elastic material · Art. 9407 Covered with elastic, antibacterial material (containing silver) · For improving pressure in the ankle area · Made of silicone; soft and flat 	Lymphpad Line · Art. 9430 <ul style="list-style-type: none"> · For padding or increasing compression · Synthetic latex, reverse side made of polyamide non-woven material · Individually incorporated into a pocket 	Lymphpad Square · Art. 9431 <ul style="list-style-type: none"> · For padding or increasing compression · Synthetic latex, reverse side made of polyamide non-woven material · Individually incorporated into a pocket


Juzo SoftCompress			
			
Malleo pad · Art. 6843 <ul style="list-style-type: none"> · For improving pressure in the ankle area · With a skin-friendly cotton cover 	Knee pad · Art. 6845 <ul style="list-style-type: none"> · Optimisation of the compression ratio at the back of the knee 	Genital pad for women · Art. 6839 <ul style="list-style-type: none"> · For insertion into a compression pantyhose · Targeted pressure optimisation in the case of genital oedema; ideal pressure distribution 	Genital pad for men · Art. 6840 <ul style="list-style-type: none"> · For insertion into a compression pantyhose · Targeted transfer of pressure and padding for the male genital area

Fastening		Care products	
			
			
Juzo Adhesive lotion · Art. 9800		Juzo Lymph lotion · Art. 9860	
<ul style="list-style-type: none"> · Special roll-on for securely holding stockings and sleeves 		<ul style="list-style-type: none"> · Moisturising lotion with 4% urea for targeted skincare in cases of lymphoedema and lipoedema 	
<ul style="list-style-type: none"> · Cooling and caring balm for smooth skin during compression therapy 		<ul style="list-style-type: none"> · Particularly mild and economical concentrated detergent for gentle cleaning of compression garments 	

Donning and doffing aids			
			
			
			
			
Juzo EasyFit XL · Art. 1890			
Juzo Slippie Leg · Art. 1901			
Arion Easy-Slide Leg · Art. 1702			
Arion Sim-Slide® · Art. 1705			
<ul style="list-style-type: none"> · Donning and doffing aid · For open/closed toes 			
<ul style="list-style-type: none"> · Donning aid made of low friction Tyvek · For open toes 			
<ul style="list-style-type: none"> · Donning aid for open toes 			
<ul style="list-style-type: none"> · Donning and doffing aid for open toes 			



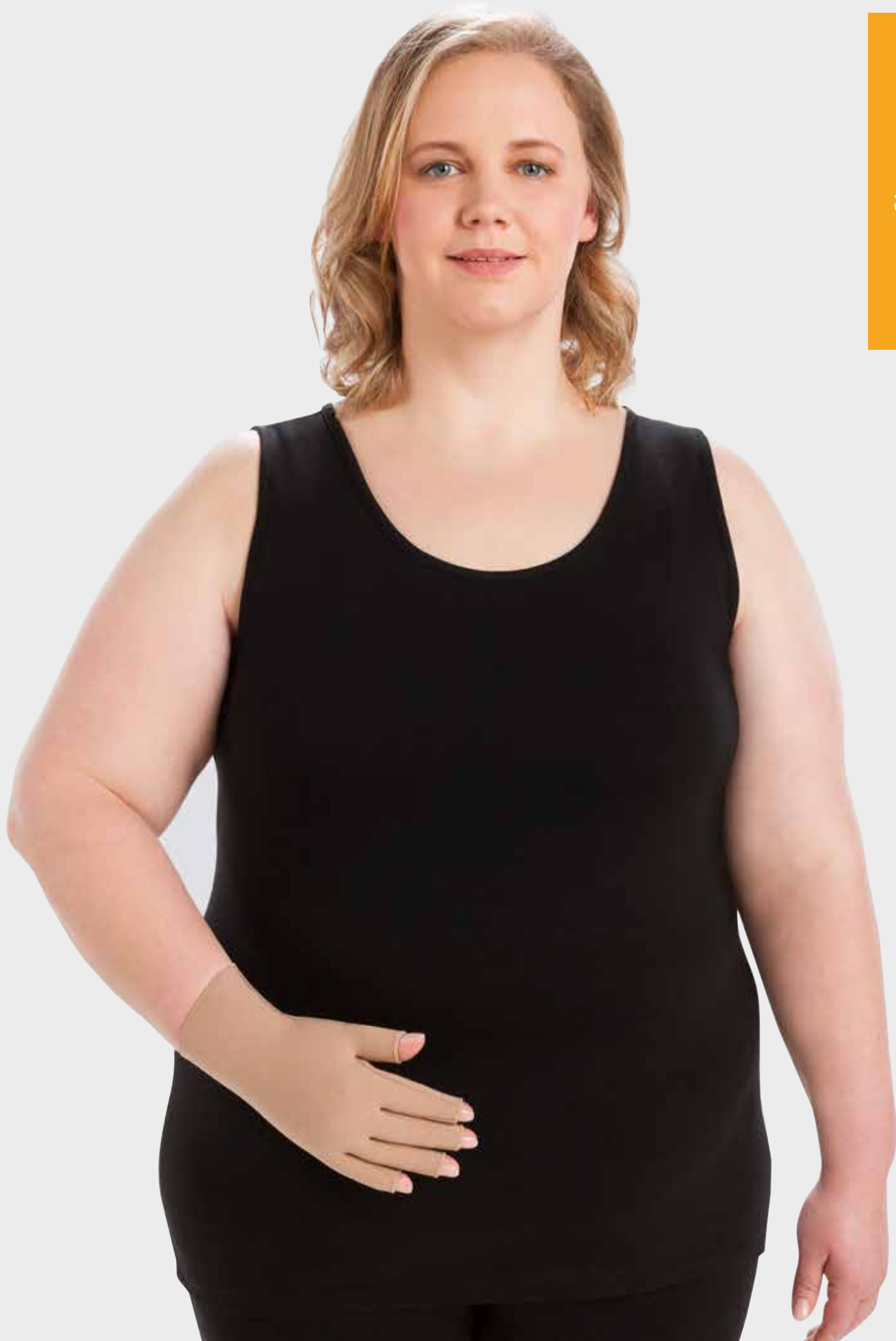


Donning and doffing aids		Additional products	Liner stocking
			
			
Arion Magnide® 2in1 · Art. 1709		Special gloves · Art. 9301	
Arion Easy-Off · Art. 1711		Juzo Liner Silver · Art. 9640	
<ul style="list-style-type: none"> · Donning and doffing aid for closed toes 		<ul style="list-style-type: none"> · Anti-slip coating · Simplifies donning of compression garments 	
<ul style="list-style-type: none"> · Doffing aid for open toes 		<ul style="list-style-type: none"> · With antibacterial silver thread · Fixation of wound dressings and pads 	









Gloves







- Available in
- Juzo Expert
 - Juzo Expert Silver
 - Juzo Expert Cotton
- Juzo Expert Strong
 - Juzo Expert Strong Silver

Glove		Model variants	
			
Glove		Long glove	
			
Finger support		One-piece sleeve and glove	







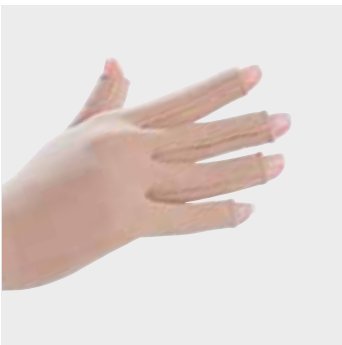



Finger			
			
Open fingers	Closed fingers	Hand portion with open thumb stub	Open finger support
		· Thumb closed also possible	· Finger support closed also possible

Border			
			
Overheight	Silicone border · 3.5 cm	Balance silicone border · 3.5 cm	Longer glove
· Lateral diagonal border · Optimised pressure discharge to proximal	· Silicone nubs ensure a high degree of stability and breathability	· Especially high breathability · Low shear forces	· Individual lengths possible up to max. "cE"

Extra options

			
Zipper	Incorporated pockets for pads	Silicone coating	Tricot lining
<ul style="list-style-type: none"> · Individual positioning 	<ul style="list-style-type: none"> · Pocket for standard pad · Customisation based on markings · Also possible with silver in case of skin irritation 	<ul style="list-style-type: none"> · Improved grip · Possible with open and closed fingers 	<ul style="list-style-type: none"> · Between the thumb and index finger · For sensitive skin · Also possible with silver in case of skin irritation
			
Seams on the outside	Closed finger support with fastener		
<ul style="list-style-type: none"> · Particularly flat seam touches the skin · Suitable for sensitive skin 	<ul style="list-style-type: none"> · With touch fastener · Open finger support also possible 		

Additional products· Gloves


Lymphpads		Pads	
			
Lymphpad Line · Art. 9430	Lymphpad Square · Art. 9431	Pressure pad · long · Art. 9420L	Pressure pad · short · Art. 9420
<ul style="list-style-type: none"> · For padding or increasing compression · Synthetic latex, reverse side made of polyamide non-woven material · Individually incorporated into a pocket 	<ul style="list-style-type: none"> · For padding or increasing compression · Synthetic latex, reverse side made of polyamide non-woven material · Individually incorporated into a pocket 	<ul style="list-style-type: none"> · Art. 9421L Covered with elastic, antibacterial material (containing silver) · Improved transfer of pressure to the back of the hand; extends across the wrist 	<ul style="list-style-type: none"> · Art. 9421 Covered with elastic, antibacterial material (containing silver) · Improved transfer of pressure to the back of the hand
Juzo SoftCompress		Zipper	
			
Hand pad · Art. 6844	Zipper extension · Art. 9532	Juzo Lymph lotion · Art. 9860	Juzo Special detergent · Art. 9000
<ul style="list-style-type: none"> · Improved transfer of pressure to the back of the hand 	<ul style="list-style-type: none"> · Easier to open and close 	<ul style="list-style-type: none"> · Moisturising lotion with 4 % urea for targeted skin care in cases of lymphoedema and lipoedema 	<ul style="list-style-type: none"> · Particularly mild and economical concentrated detergent for gentle cleaning of compression garments
Donning aids		Additional products	
			
Arion Dactyna · Art. 1708	Special gloves · Art. 9301		
<ul style="list-style-type: none"> · For use with compression gloves with open fingers 	<ul style="list-style-type: none"> · Anti-slip coating · Simplifies donning of compression garments 		




Sleeves

- Available in
- Juzo Expert
 - Juzo Expert Silver
 - Juzo Expert Cotton
 - Juzo Expert Strong
 - Juzo Expert Strong Silver


CG




Model variants




Sleeves (CG) with overheight



Shoulder and strap




One-piece sleeve and glove




Forearm cuff (CE)

Border · Fastening




Overheight / max. overheight

- Lateral diagonal border
- Optimised pressure discharge to proximal




Top border

- Additional fastening necessary (e.g. adhesive lotion)




Silicone border · 3.5 cm
Silicone border pattern · 3.5 cm

- Silicone border: 5 cm also possible
- Silicone nubs ensure a high degree of stability and breathability




Balance silicone border · 3.5 cm
Balance silicone border pattern · 3.5 cm

- Balance silicone border: 5 cm also possible
- Especially high breathability
- Low shear forces




Elastic border 3.5 cm

- Stable border
- Silicone-free




Bra fastening

- Fastening option with touch fastener at the bra strap



Shoulder and strap

- Alternative to silicone border

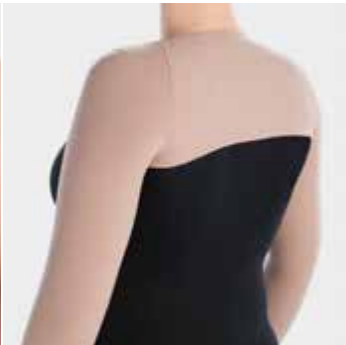

Bolero connection

- Fastening option in the case of bilateral arm support
- With long and/or short sleeves
- Back portion without compression

Extra options

**Anatomically flexed form at "cE"**

- 30° or 50° angulation possible
- In the case of a predominately angled elbow position

**Seam at the outside of the arm with CG**

- In the case of sensitivity in the crook of the elbow
- Only possible with anatomically angled shape 30° and 50°

**Adhesive borders**

- Additional adhesion on the upper arm
- Possible positioning: lateral at the outer side

**Sewn-in silicone border on the inside**

- Only with overheight
- Increased stability and adhesion
- 3/4 sewn-in on the inside also possible
- Can be combined with silicone border at the top

**Bra fastening with silicone support**


- Additional hold support in the shoulder area

**Bolero connection with silicone support**

- Additional hold support in the shoulder area

**Tricot lining at "cE"**

- In the case of sensitive skin in the crook of the elbow
- Also possible with silver in case of skin irritation

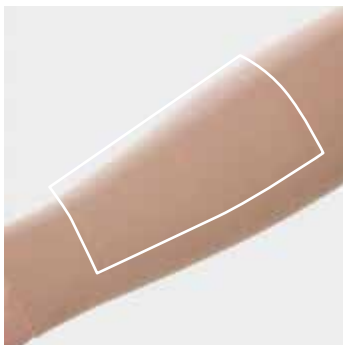**Incorporated pockets for pads**

- Customisation based on markings
- Also possible with silver in case of skin irritation









**Seams on the outside**

- Particularly flat seam touches the skin
- Suitable for sensitive skin

**Zipper**

- Individual positioning

Additional products · Sleeves




Lymphpads		Juzo SoftCompress	Fastening
			
Lymphpad Line · Art. 9430	Lymphpad Square · Art. 9431	Elbow pad · Art. 6842	Juzo Adhesive lotion · Art. 9800
<ul style="list-style-type: none"> · For padding or increasing compression · Synthetic latex, reverse side made of polyamide non-woven material · Individually incorporated into a pocket 	<ul style="list-style-type: none"> · For padding or increasing compression · Synthetic latex, reverse side made of polyamide non-woven material · Individually incorporated into a pocket 	<ul style="list-style-type: none"> · Optimization of the compression ratio in the crook of the elbow 	<ul style="list-style-type: none"> · Special roll-on for securely holding stockings and sleeves
Care products		Donning aids	Additional products
			
Juzo Lymph lotion · Art. 9860	Juzo Special detergent · Art. 9000	Arion EasySlide Arm · Art. 1701	Special gloves · Art. 9301
<ul style="list-style-type: none"> · Moisturising lotion with 4% urea for targeted skincare in cases of lymphoedema and lipoedema 	<ul style="list-style-type: none"> · Particularly mild and economical concentrated detergent for gentle cleaning of compression garments 	<ul style="list-style-type: none"> · Smooth, double-layered material 	<ul style="list-style-type: none"> · Anti-slip coating · Simplifies donning of compression garments




Thorax garments

Available in

- Juzo Expert
- Juzo Expert Silver

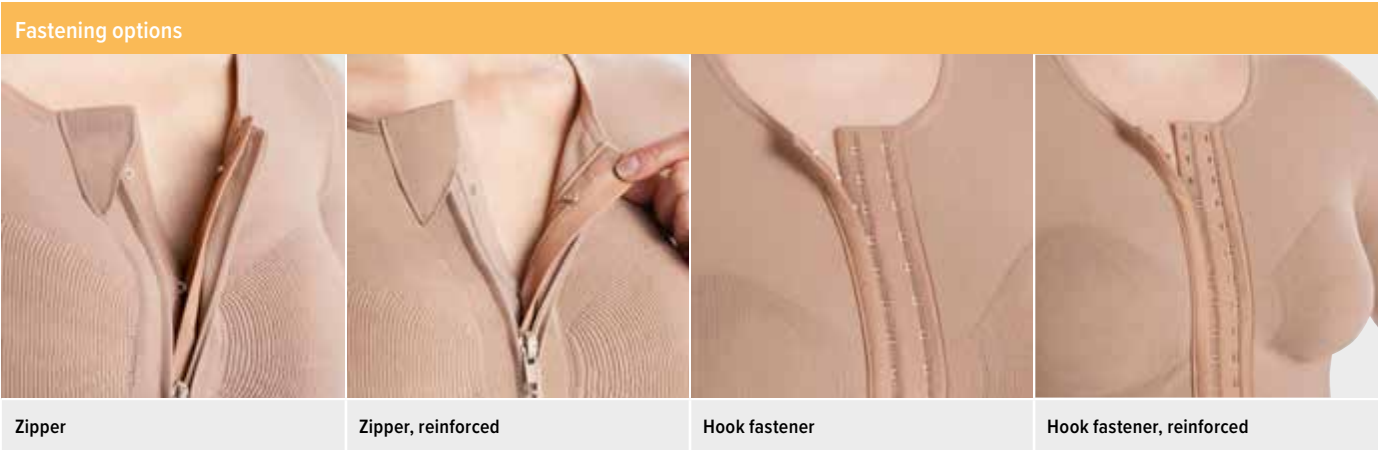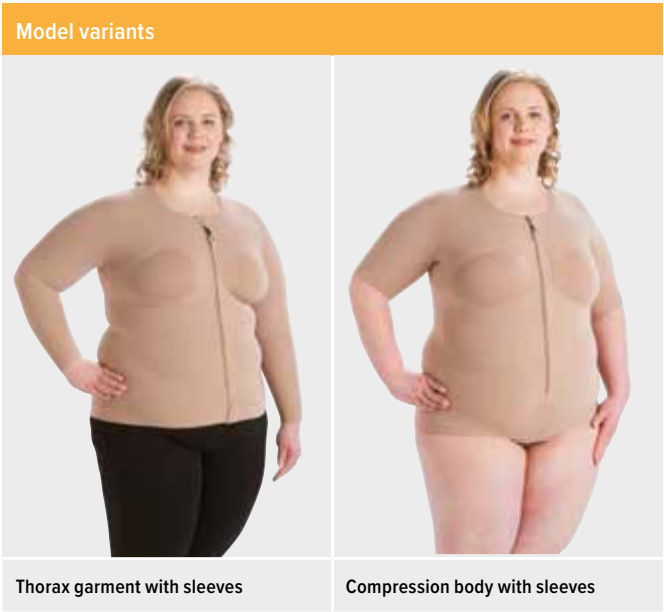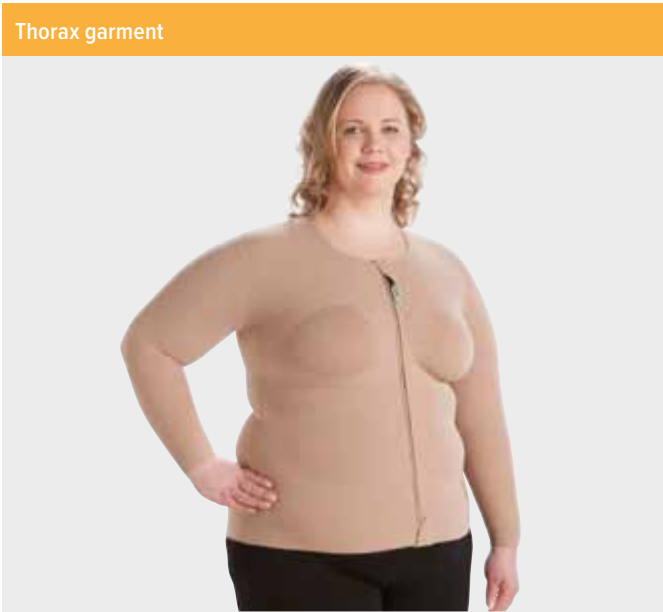

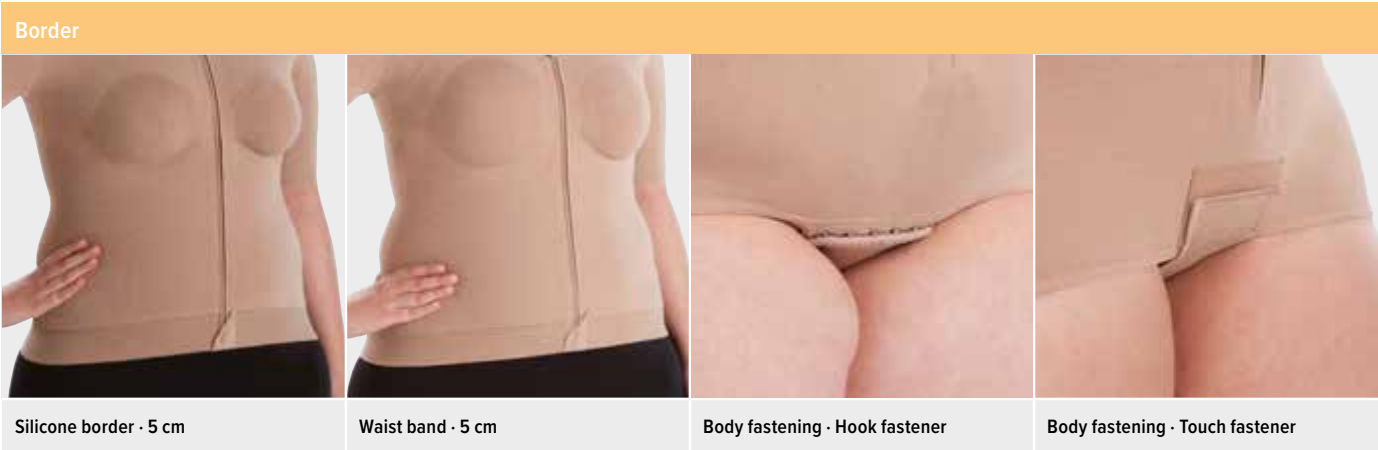


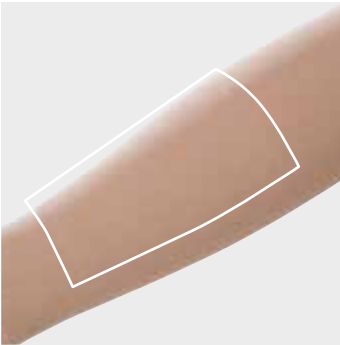





Zipper	Zipper, reinforced	Hook fastener	Hook fastener, reinforced
<ul style="list-style-type: none">· With hook / eye underlap for easier closure· Individual positioning is possible	<ul style="list-style-type: none">· With reinforced hook / eye underlap· Greater resilience for more extensive use· Individual positioning is possible	<ul style="list-style-type: none">· Second row of loops enables adjustment· Individual positioning is possible	<ul style="list-style-type: none">· Second row of loops enables adjustment· Greater resilience for more extensive use· Individual positioning is possible











Silicone border · 5 cm	Waist band · 5 cm	Body fastening · Hook fastener	Body fastening · Touch fastener
<ul style="list-style-type: none">· Flat, distal border· With silicone nubs	<ul style="list-style-type: none">· Flat, elastic, distal border	<ul style="list-style-type: none">· Prevents riding up	<ul style="list-style-type: none">· Prevents riding up· Alternative to hook fastener

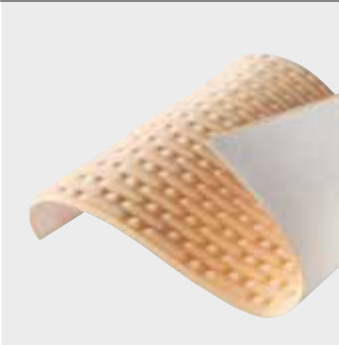

Extra options

			
Knitted-in bra cups	Prosthetics pocket	Anatomically flexed form at "cE"	Tricot lining at "cE"
<ul style="list-style-type: none"> · Seamless bra cup · Lightly compressive 	<ul style="list-style-type: none"> · Sewn-in prosthetics pocket with three openings 	<ul style="list-style-type: none"> · 30° or 50° angulation possible · In the case of a predominately angled elbow position 	<ul style="list-style-type: none"> · In the case of sensitive skin in the crook of the elbow · Also possible with silver in case of skin irritation
			
Arm portion with integrated zipper	Incorporated pockets for pads	Stand-up collar	Seams on the outside
<ul style="list-style-type: none"> · Individual positioning 	<ul style="list-style-type: none"> · Customisation based on markings · Also possible with silver in case of skin irritation 	<ul style="list-style-type: none"> · In the case of oedema in the throat and neck regions or in the upper shoulder area (recommended in combination with a head bandage depending on the circumstances) 	<ul style="list-style-type: none"> · Particularly flat seam touches the skin · Suitable for sensitive skin
			
Inelastic back zone			
<ul style="list-style-type: none"> · For adults · Promotes upright body posture · Prevents a relief posture from being adopted 			

Additional products · Thorax garments

Lymphpads		ScarPad	
			
Lymphpad Line · Art. 9430	Lymphpad Square · Art. 9431	ScarPad Extra · Art. 9411	ScarPad Extra Strong · Art. 9412
<ul style="list-style-type: none"> · For padding or increasing compression · Synthetic latex, reverse side made of polyamide non-woven material · Individually incorporated into a pocket 	<ul style="list-style-type: none"> · For padding or increasing compression · Synthetic latex, reverse side made of polyamide non-woven material · Individually incorporated into a pocket 	<ul style="list-style-type: none"> · Silicone patch for scars 	<ul style="list-style-type: none"> · Silicone patch for scars

Juzo SoftCompress		Zipper	Care products
			
Breast pad · Art. 6837	Elbow pad · Art. 6842	Zipper extension · Art. 9532	Juzo Special detergent · Art. 9000
<ul style="list-style-type: none"> · For women · Targeted transfer of pressure in the breast and thorax area 	<ul style="list-style-type: none"> · Optimisation of the compression ratio in the crook of the elbow 	<ul style="list-style-type: none"> · Easier to open and close 	<ul style="list-style-type: none"> · Particularly mild and economical concentrated detergent for gentle cleaning of compression garments



Donning aids	Additional products
	
Arion EasySlide Arm · Art. 1701	Special gloves · Art. 9301
<ul style="list-style-type: none"> · Smooth, double-layered material 	<ul style="list-style-type: none"> · Anti-slip coating · Simplifies donning of compression garments




Head garments




Available in

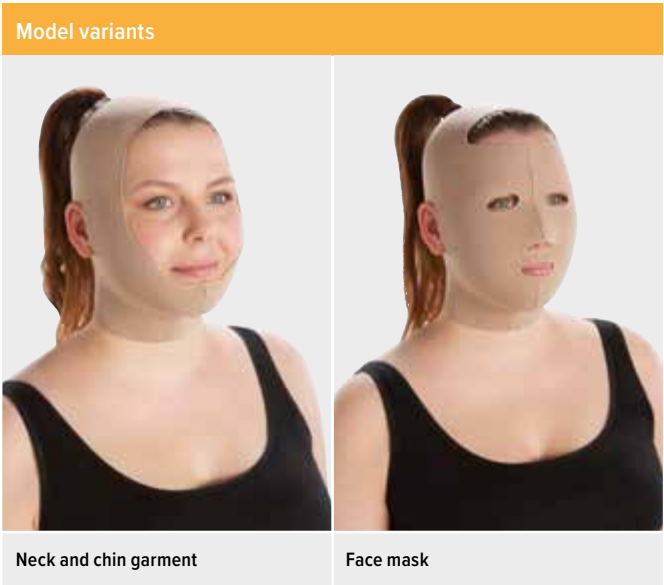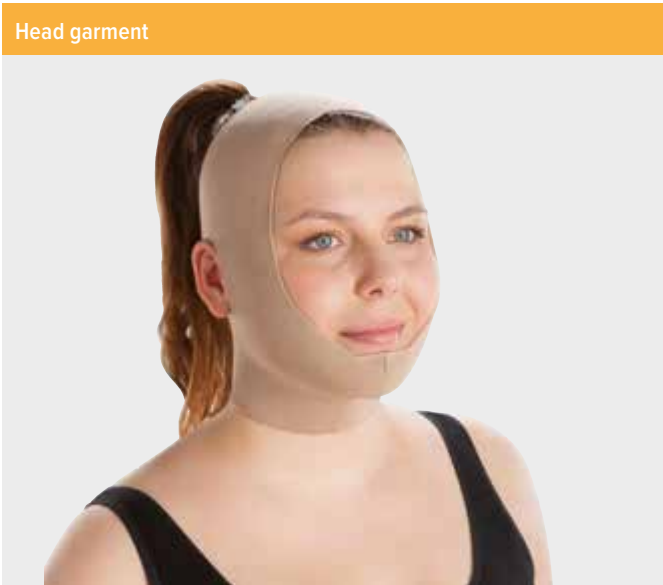- Juzo Expert
- Juzo Expert Silver

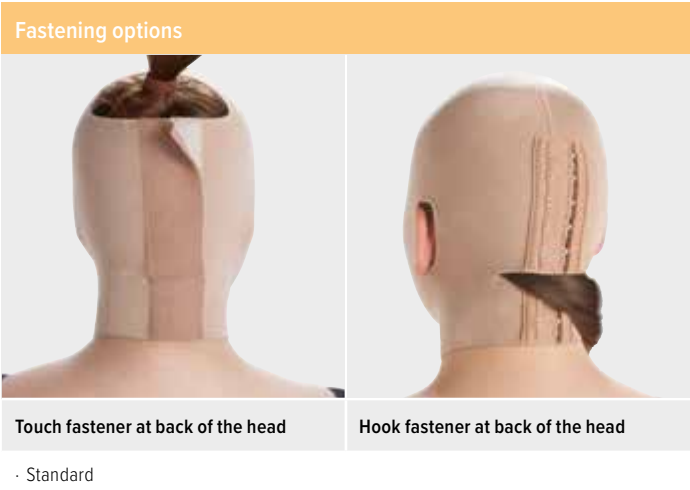

Head			
Face mask Closed forehead and back of the head	Face mask Open forehead and back of the head	Face mask Open forehead and back of the head	Neck and chin garment Open forehead and back of the head
<ul style="list-style-type: none">· Cut-outs possible for eyes, nose, mouth, ears· Knitted-in nose possible	<ul style="list-style-type: none">· Cut-outs possible for eyes, nose, mouth, ears· Knitted-in nose possible	<ul style="list-style-type: none">· Free facial area· Cut-outs possible for ears	<ul style="list-style-type: none">· Cut-outs possible for ears



Extra options		
		
Openings		Knitted-in nose
<ul style="list-style-type: none"> · Possible for eyes, mouth, ears and nose 		<ul style="list-style-type: none"> · Compression in the nose area
		
NEW Tracheostoma opening		
<ul style="list-style-type: none"> · Customisation based on markings 		

Additional products · Head garments

Lymphpads		ScarPad
		
Lymphpad Line · Art. 9430	Lymphpad Square · Art. 9431	ScarPad Light · Art. 9410
<ul style="list-style-type: none"> · For padding or increasing compression · Synthetic latex, reverse side made of polyamide non-woven material · Individually incorporated into a pocket 		<ul style="list-style-type: none"> · Silicone patch for scars





- Available in
- Juzo Expert
 - Juzo Expert Silver
 - Juzo Expert Cotton


Stump garments (arm and leg)



Model variants

- Below-knee stump
- Below-knee stump after Pirogoff amputation
- Above-knee stump
- Arm stump

Border/Leg fastening

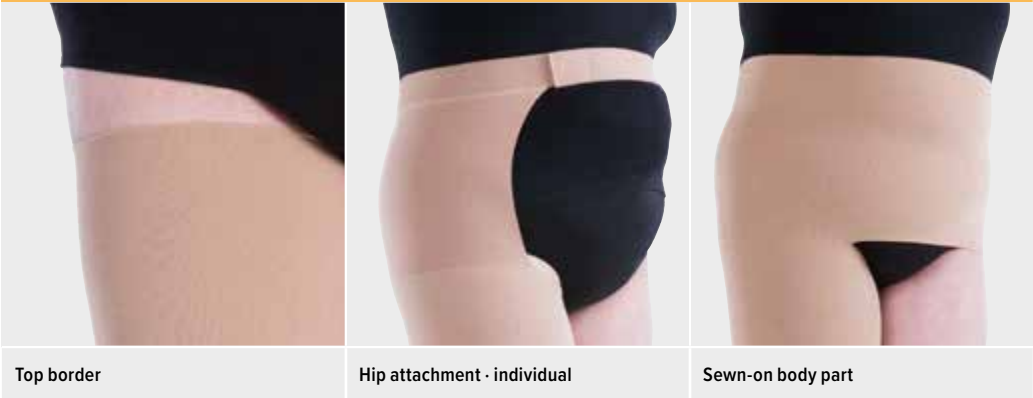

Top border

- Additional fastening necessary (e.g. adhesive lotion)

Hip attachment · individual

- Additional silicone border for better fixation is possible
- Worn on the left or right

Sewn-on body part

- Alternative fastening option with compression in the stomach area

Border/Arm fastening



Top border



- Additional fastening necessary (e.g. adhesive lotion)

Bra fastening







- Fastening option with touch fastener at the bra strap

Shoulder and strap

- Alternative to silicone border

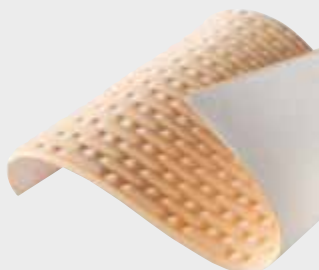Silicone borders		Additional products
		
Silicone border · 3.5 cm/5 cm	Balance silicone border · 3.5 cm/5 cm	Prosthesis pin opening · 2 cm
<ul style="list-style-type: none"> · 3.5 cm can be used with below-knee and arm stump shrinkers · 5 cm can be used with below-knee and above-knee stump shrinkers · Silicone nubs ensure a high degree of stability and breathability 	<ul style="list-style-type: none"> · 3.5 cm can be used with below-knee and arm stump shrinkers · 5 cm can be used with below-knee and above-knee stump shrinkers · Especially high breathability · Low shear forces 	<ul style="list-style-type: none"> · 2 cm opening to the end of the stump

Additional products · Stump garments (arm and leg)

Lymphpads		ScarPad	
			
Lymphpad Line · Art. 9430	Lymphpad Square · Art. 9431	ScarPad Extra · Art. 9411	ScarPad Extra Strong · Art. 9412
<ul style="list-style-type: none"> · For padding or increasing compression · Synthetic latex, reverse side made of polyamide non-woven material · Individually incorporated into a pocket 	<ul style="list-style-type: none"> · For padding or increasing compression · Synthetic latex, reverse side made of polyamide non-woven material · Individually incorporated into a pocket 	<ul style="list-style-type: none"> · Silicone patch for scars 	<ul style="list-style-type: none"> · Silicone patch for scars
Care products		Additional products	
			
Juzo Special Detergent · Art. 9000	Special gloves · Art. 9301		
<ul style="list-style-type: none"> · Particularly mild and economical concentrated detergent for gentle cleaning of compression garments 	<ul style="list-style-type: none"> · Anti-slip coating · Simplifies donning of compression garments 		

Diverse combination options

Multiple-part garments

Multiple-part garments are required in order for patients with comorbidities such as rheumatism, back complaints or incontinence to put on and take off the garments easier.

Common combination options

- AD + Capri (BT, B¹T, CT)
- AG + Bermuda (ET, FT)
- Sleeve (CG) with glove

Benefits

- Easy to put on and take off
- Well-fitting garment for “changes in diameter”

Double coverage

If an increase in local pressure is required, you can use two layers of stockings. For example, the combination of below-knee compression stockings and leggings is recommended for distally pronounced lymphoedema of the lower extremities. Conversely, using two layers of stockings with thigh-high stockings and bermuda pants may be necessary if, for example, there is a more proximal emphasis in lipoedema or secondary lymphoedema. It should be noted that, when using two layers of garments in this way, the pressure values add up, but the compression class does not double. For example, two 23 mmHg (CCL 2) garments combined produce a total of 46 mmHg, which corresponds to CCL 3.

Common combination options

- AD + Capri (BT, B¹T, CT)
- AG + Bermuda (ET, FT)
- AG + Capri (BT, B¹T, CT)
- Sleeve (CG) + forearm sleeve with hand piece

Benefits

- Targeted increase in pressure in the overlapping areas
- Combination of different flat-knit products possible
- Combination of different compression classes possible

Examples for multiple-part garments



AD-stocking + Capri (B¹T)



AG-stocking + Bermuda (ET)

Practical tips for caregivers

Check the fit regularly

Compression garments must fit perfectly. However, the way a garment fits can change over time if the patient gains or loses weight, or if circumferences change (e.g. oedema reduction).

Caregivers should therefore regularly check to make sure garments still fit properly and then have alterations made or recommend that a new garment be produced if necessary.

Explain how to take care of the product

Compression garments should be washed every day at 40 °C in order to allow the material to regenerate. It is recommended that garments be washed with Juzo Special Detergent · Art. 9000 and that a fabric softener not be used.

Get patients to take care of their skin daily

Dry skin can be kept supple by massaging it and applying a cream every day. Skin should always be kept clean and elastic in order to prevent inflammation.

Explain the wearing period

Compression garments should be worn as prescribed by a physician, as this is the only way to guarantee an optimum treatment outcome. Garments must be worn consistently every day or for the periods prescribed by a physician.

In order to ensure proper hygiene and material regeneration, caregivers should instruct patients to change their compression garments every day. Ideally, patients should therefore be issued more than one compression garment.

Instruct patients to avoid excessive exposure to sunlight

Exposure to direct sunlight should be avoided in order to prevent skin changes and pigment disorders.

Case study

Compression therapy with Juzo Expert and Juzo Expert Strong

Ursula K. (born in 1954) was diagnosed with cervical cancer at the age of 39. Following a Wertheim-Meigs operation and the removal of more than 30 lymph nodes and a lymph cyst in her lower abdomen, Ursula K. quickly developed lymphoedema in both legs. She has been receiving treatment for this in the form of MLD and compression therapy since 1993. She was initially given a circular knit, but the results were less than optimal. Ursula K. came across Juzo Expert by coincidence after about three years of treatment. Since that time, she has been wearing Juzo Expert and Juzo Expert Strong.



Secondary lymphoedema in both legs following several surgeries to address cervical cancer



Treatment example: Pantyhose (AT), Juzo Expert



Treatment example – multiple products: Stockings (AG), left leg in Juzo Expert Strong (CCL 3), right leg in Juzo Expert (CCL 2), with Juzo Expert leggings (CCL 2) on top

Juzo Customer Service International

Phone: +49 (0)8251 901 300 · E-mail: order@juzo.com · Shop: portal.juzo.com



Julius Zorn GmbH · Juliusplatz 1 · 86551 Aichach · Germany · juzo.com